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

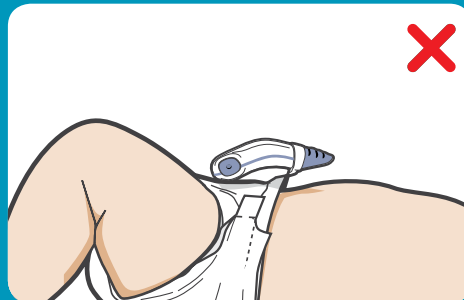
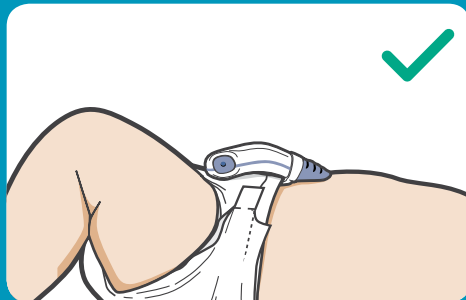
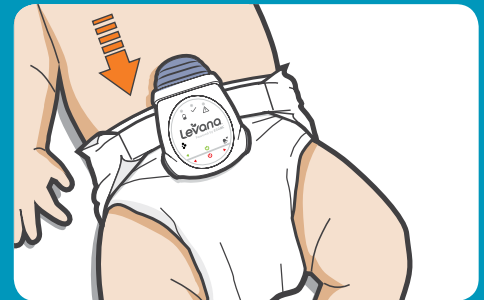
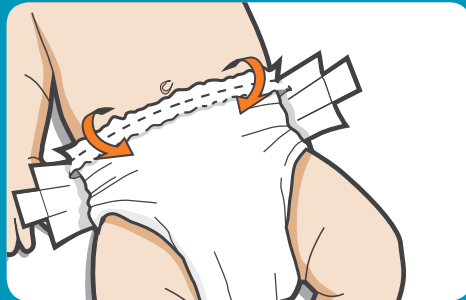
Ensure that Oma+ sits squarely on the diaper, and that the Movement Sensor is in contact with baby's tummy.

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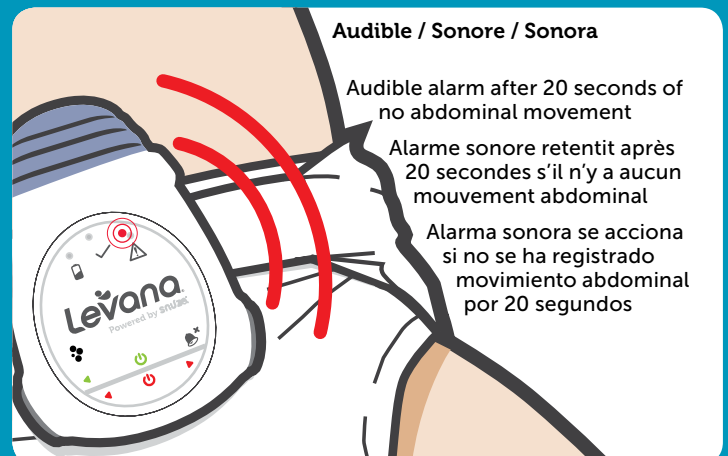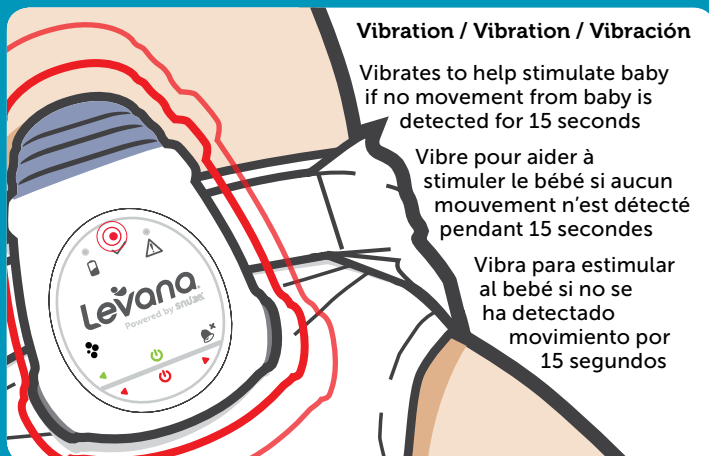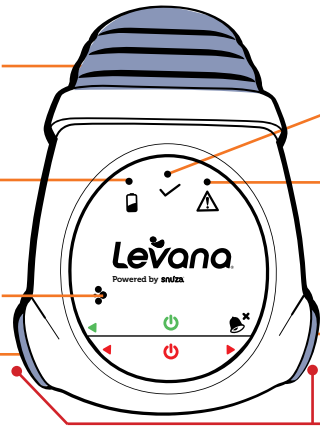
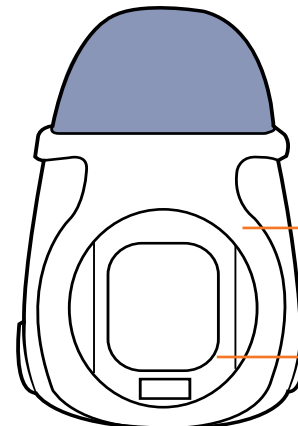
DO NOT RETURN THIS PRODUCT TO THE STORE!

Contact Customer Support anytime for assistance.

Product warranty information

We take quality very seriously. This is why all of our LEVANA® products come with a one-year warranty from the original purchase date against defects in workmanship and materials. If you have warranty or support issues please contact us using any of the following methods:

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LEVANA Canada
4080 Montrose
Road
Niagara Falls, ON
Canada L2H 1J9

LEVANA USA
60 Industrial
Parkway, Suite #Z64
Cheektowaga, NY
USA 14227

- Only use monitor when baby is in a motionless bed/cot. Do not use in car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.
- This product is a baby care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap including SIDS (Sudden Infant Death Syndrome).
- This product does not replace responsible adult supervision. Check on your child at regular intervals and ensure that the battery is full.
- The replaceable battery is designed to last for a 12 month period, being used on average 6 times per day for a total of 18 hours per day, allowing up to 4 stimulations per day and 1 alarm per day for no longer than 20 seconds.
- SNUZA is a registered trademark of Biosentronics CC.
- ©2013 LEVANA®. All rights reserved. LEVANA®, the LEVANA® logo, Oma+™ and other LEVANA® marks may be registered. All other trademarks are the property of their respective owners.

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