**INSTRUCTIONS**

**FOR YOUR DENTAL**

**IMPRESSIONS**

**smileunion**

Deutsch • English • Español • Italiano • Français • Nederlands

<b>Deutsch</b>	<b>4-17</b>
<b>English</b>	<b>18-31</b>
<b>Español</b>	<b>32-45</b>
<b>Italiano</b>	<b>46-59</b>
<b>Français</b>	<b>60-73</b>
<b>Nederlands</b>	<b>74-87</b>

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English

Please read our instructions carefully. Before returning the impressions, write your name on this page and enclose it with the impressions.

## Important

Name:

Date of birth:

Please include these instructions when you return your impressions so that we can match them correctly.

# How do I take the necessary impressions of my jaw?

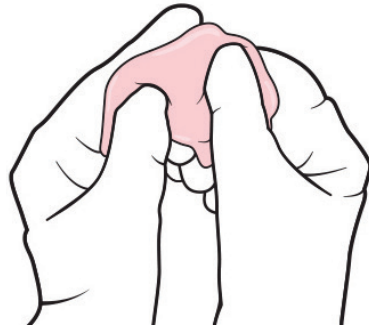
**Before you start, you should prepare the following things for a smooth process:**

- Ideally, we need four impressions of your jaw (2x upper jaw, 2x lower jaw). You have material for a total of six attempts.
- Total time required is approx. 30 minutes.
- Place all the materials from the smileunion impression set in front of you on a table.
- Have a mirror ready, if you have one.
- Use your smartphone as a timer.
- Clean your hands.
- Before you start, ideally watch our instructional video at [www.smileunion.de](http://www.smileunion.de).
- Brush your teeth thoroughly. This is the only way to get the perfect impression, which is essential for your treatment.
- Take your time and read through the following steps in advance.

# 01

## Taking impressions of the jaw

*(until minute 00:45)*


The gloves provided can be used optionally. Take one blue and one white impression compound from the enclosed tins. Let's go - start the timer!

Mix the two impression compounds and knead them vigorously and evenly until you get a uniform blue colour without streaks. You have 45 seconds for this step.

# 01
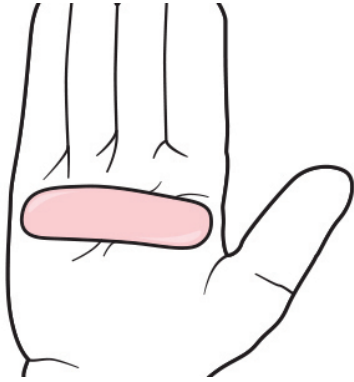
# 02


# 03

*(until minute 00:55)*

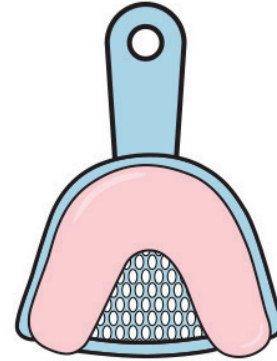Now roll the homogeneous impression compound between your palms within the next 10 seconds until it is approx. 8-12cm long. It should no longer have any white streaks.

*(until minute 01:15)*


Now place the roll in the prepared impression tray so that the impression compound fills the tray. Try to leave some space around the edges.

# 02
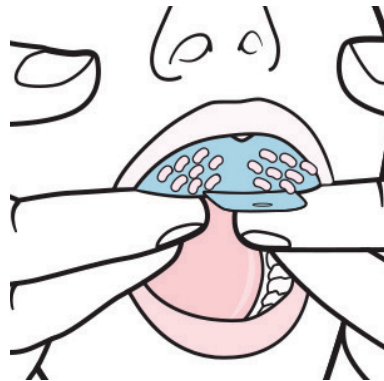
# 03

# 04

(until minute 04:15)

Place the impression tray in your mouth. Note: The handle of the impression tray should be centred on your nose. Now press the impression tray evenly with your two thumbs so that your anterior and posterior teeth are imprinted at a uniform depth.

The impression should stay in your mouth for **at least 3 minutes**. The impression should not be moved in order to ensure that the impression compound obtains the necessary strength.

**Note:** The impression compound must be pressed firmly and evenly over the teeth so that the transition from the teeth to the gums is clearly visible in the impression.

# 04

# 05

# 06

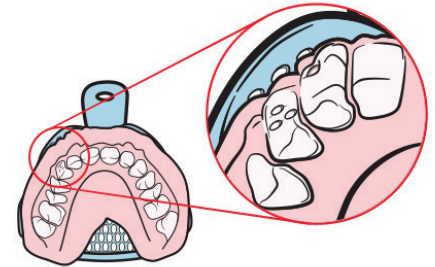
Almost there! Now it's time to stay calm. Now you just have to remove the impression carefully. Caution: Try to remove it slowly and with tilting movements. Sometimes it takes a little bit longer - take your time.

Do you think the impression came out well? Check it again to see if it has any flaws. If you don't want to use it, continue with step 7.

Did everything go well?  
Then go to step 8.

# 05

# 06

# 07 08

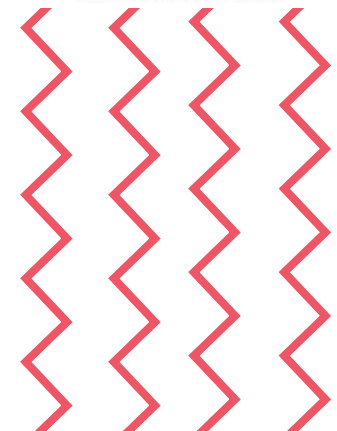
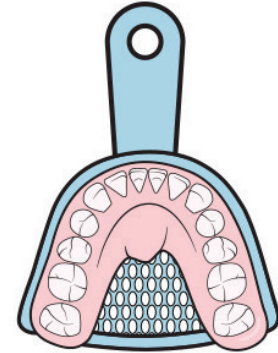
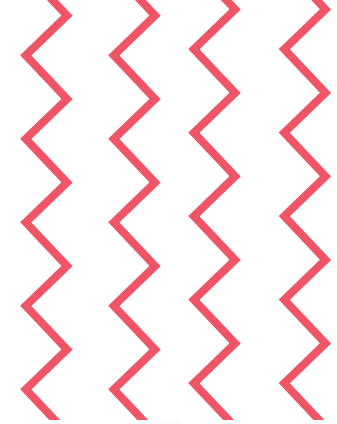
Be aware of possible sources of error, remove the “bad” impression from the impression tray and try again. Use two new pots with blue and white impression compound. Take your time to read the instructions again.

**Important:** Do NOT release the impression compound from the impression tray. Place the impression in the shipping bag. Then go back to step 1 and repeat the steps for your other 3 impressions (2 x upper jaw/2 x lower jaw).

Once you have your 4 impressions in the shipping bag, send them off in the original shipping box within 2-3 days.

We have already included a return shipping label for you.

Please do not forget to send us the photos of your jaws (see next page). Even if we have already received your impressions, we cannot create your treatment plan until we have received the pictures.

# How do I take the necessary photos of my teeth and my malocclusion?

Take the supplied cheek retractor (OptraGate) and insert it. You can take the photos with your mobile phone - activate the flash. Send us the pictures you have taken, stating your name and date of birth, by email to [labor@smileunion.de](mailto:labor@smileunion.de) or by WhatsApp to +49 (0) 1579 2453 234.

## Frontal view (closed)

The cheek retractor (OptraGate) must fit as shown. Your teeth should be completely visible without being covered by your upper/lower lip. Bite firmly.

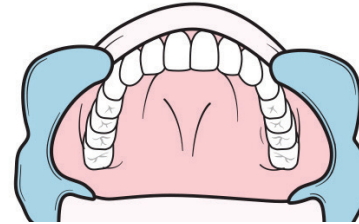
01

## Frontal view

*(slightly open)*

The cheek retractor must fit as shown. Your teeth should be completely visible without being covered by the upper/lower lip. Open your mouth slightly until a small gap is visible between the upper and lower teeth.


## Upper jaw guard

With the cheek retractor in your mouth, open your mouth wide to photograph the upper teeth.

02

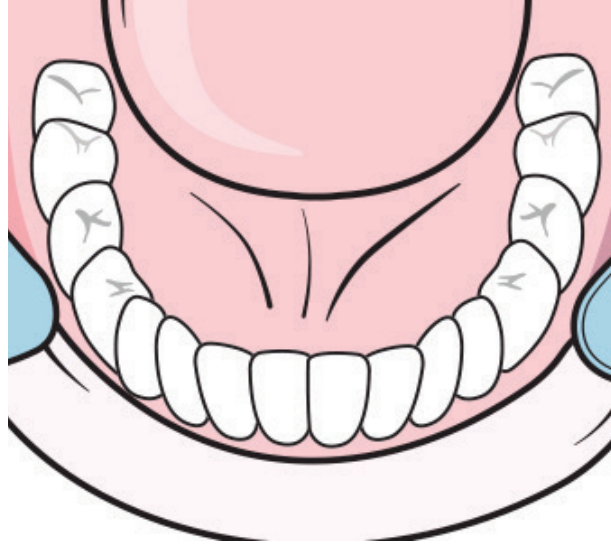03


## Lower jaw view

With the cheek retractor in your mouth, open your mouth wide again to photograph your lower teeth as shown.

04

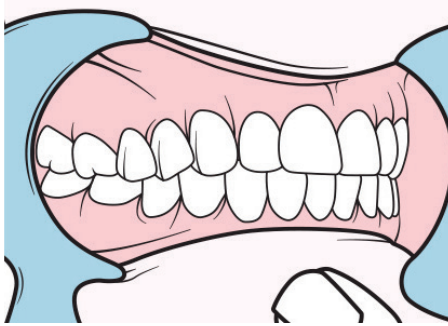
04

## **Laterally closed** *(right side)*

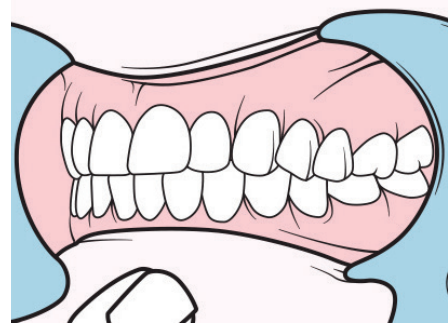To get a good lateral view, you can pull the cheek retractor slightly backwards with your respective hand. All posterior teeth should be visible.


## **Laterally closed** *(left side)*

To get a good lateral view, you can pull the cheek retractor slightly backwards with your respective hand. All posterior teeth should be visible.

05 06

# What happens next? Your treatment plan

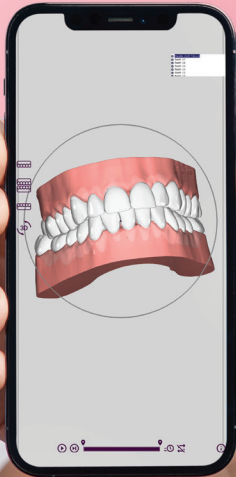- After you have sent us your impressions and photos, we will check whether your malocclusion is suitable for treatment with transparent aligners.
- If this is the case, you will receive your individual treatment plan with costs by email.
- The treatment plan contains a 3D simulation and shows you your teeth before, during and after the treatment.
- Only then do you decide whether you want to have the treatment and how you want to pay (one-time payment or monthly instalments).
- As soon as you have completed the payment process, your aligners will go into production and be sent to your home within 14 days.

## Wearing and caring for your aligners

- Always make sure that you use the correct aligner for your upper and lower jaw. Always insert each aligner separately. Don't clench your teeth to get the aligners in place - this can damage them.
- Wear your aligners as we recommend for 22 hours a day.
- Take the aligners out before eating and drinking. You can drink cold water even with the aligners in place.
- Rinse the aligners with water immediately after removing them. Store them in the protective container provided.
- Clean the aligners before putting them back in. It is best to use a soft toothbrush and toothpaste as well as cold or lukewarm water. Rinse the aligners with water after cleaning.
- Do not use denture cleaner or mouthwash. This could dull the aligners, making them more visible.
- After each meal or snack, brush your teeth before putting the aligners back in.

A young woman with long, wavy blonde hair and blue eyes is smiling broadly. She is holding a blue, U-shaped dental mold over her right eye. The mold has several small, circular indentations along its inner edge. She is wearing a light-colored, possibly white, top. The background is a solid, light blue color.

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