




Dimension	16.5" x 15" x 1.3" (42cm x 38cm x 3.2cm)
Cover Material	Black mesh fabric cover with non-slip bottom 
Seat Material	Patented non-liquid gel
Weight	1.73kg
Weight Capacity	100kg

Features

- Innovative patented diamond shaped air cell technology evenly distributes body weight to relieve pressure on coccyx, lower back and perineum
- Patented non-liquid gel seat cushion is durable due to its high resilience, high strength and flexibility nature
- Open cell column structure design dissipates heat well
- Removable seat cover and gel seat is washable
- Suitable for wheelchairs, chairs, couches, sofas, recliners and beds

Pressure Mapping Before and After Used

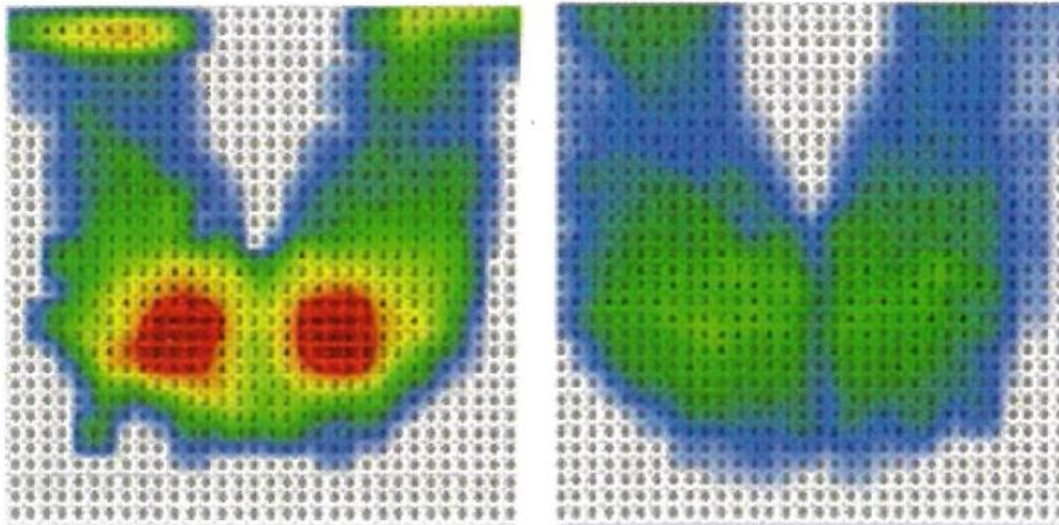

(Pic) Before use seat cushion

(Pic) After use seat cushion

Before: High pressure observed in red area before use of seat cushion

After: The seat cushion effectively reduces and dissipates pressure while providing maximum weight distribution and stability which helps prevent sores and ulcers

Cautions: Do not wash seat cushion foam/gel in washing machine. Do not use strong cleaning agent to clean the seat cover. Do not poke the seat with sharp objects

Washing Instruction:

- Remove the seat cover and hand wash it using mild detergent with cold water (normal temperature 30°C or less)
- Do not squeeze the seat cover. Allow it to dry naturally by laying on flat surface