

NAH-207

Lightweight travel chair



Folded

Super
Lightweight
6 kg

Weight
Capacity
120 kg



Description


- The lightweight travel chair is the lightest and most compactable of transport Wheelchairs.
- Great for use while traveling. The thigh belt ensures balanced seating and the flip-up armrest facilitates smoother transfers to and from the chair's seat.
- At a weight of only 6kg, the Travel Chair is not only ultra light but easy to use with quick folding and one touch opening.



Carry bag

It is convenient to carry and store.
Can be used on the plane.

SIZE

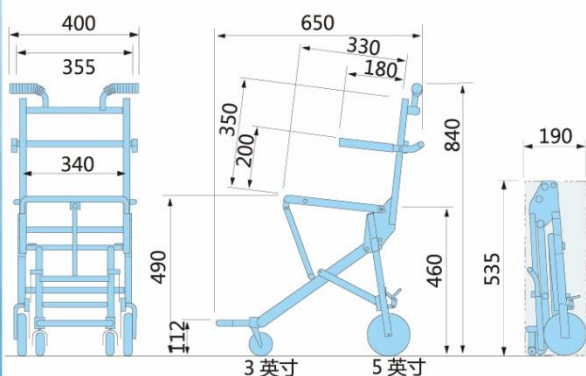

Weight	about 6 kg
Seat width	36cm/ 40cm (customized)
Size (L×W×H)	65cm×40cm×84cm
Folding size (L×W×H)	19cm×40cm×53.5cm
Height of armrest (from seat)	20cm
Height of back cushion (from seat)	40cm
Front seat height	49cm
Back seat height	46cm
Seat depth	33cm
Bearing capacity	120kg

The Folding and Unfolding Instructions of the Travel Chair


Please strictly follow the steps below.

Otherwise, it may result in hand squeezing or improper folding.



Unfolding Instructions

Firstly, make sure the front casters remain parallel to the wheels. Then hold the upper back pipe and the bumper pipe with both hands, gently pull open to the utmost until the bilateral slide pins enter into the holes and lock in.

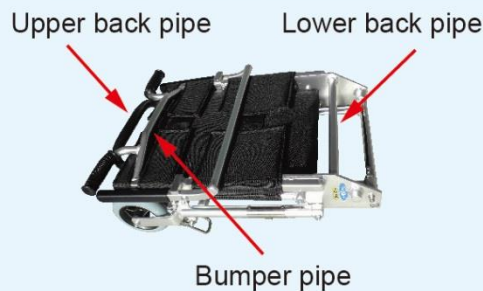

Attention: Once the slide pins enter into the holes, it will make the clicking sound. Please check if the slide pins come out or not.

Secondly,

Hold the forepart of the armrest and push it down.

Attention: please keep hands away from the armrest joint part.



Folding Instructions

Firstly, Hold the upper back pipe with one hand and the lower back pipe as well as the brake wire with another hand, then push the backrest forward and compress the outer frame tightly to fold.

The folded wheelchair must be held while moving. If toppled over, the wheelchair shall not be pushed forcibly.

