



Model: 50/JBS181-19

Overall Width:	29" (74 cm)	Seat Width:	20" (51 cm)
Seat Depth:	19" (49 cm)	Overall Depth (Fully Tilt & Recline):	68" (173 cm)
Floor to Seat Height:	20" (51 cm)	Overall Depth (Upright):	45" (114 cm)
Backrest Height:	31" (79 cm)	Total Height (Upright):	43" (109 cm)
Total Height (Fully Tilt & Recline):	35" (89 cm)	Armrest Height:	9" (23 cm)
Armrest Size:	4.17" x 34.3" (11 x 88 cm)	Tray Size:	25.5" x 15" (65 x 38 cm)
Product Weight:	49.5 kg	Max Weight Capacity:	120 kg
Country of Origin:	Taiwan		

MAINTAINENSE & CLEANING

EVERY THREE MONTHS:

Check and Clean Casters • Tighten Nuts • Check Brakes on Casters • Check and Clean Lock Recline Mechanism

For upholstery maintenance and stain removal, the following hints will be helpful:

Ordinary Dirt: Wash with warm water and a mild soap detergent.

Stubborn Dirt: Using a soft bristle brush, follow directions above.

Ground-in Dirt: Scrub with powdered cleanser or similar detergent.

Chewing Gum: Scrape carefully; remove with kerosene or naphtha.

Nail Polish and Remover: Blot immediately to prevent damage.

Tars, Asphalt, Creosote: Remove to prevent staining. Clean area with kerosene or naphtha.

Caution: When using kerosene or naphtha, do so in a well ventilated area and away from fire and flame.

Paint, Shoe Polish, Heel Marks: Remove immediately! Use a white cloth dampened in kerosene, naphtha or turpentine. Do not use paint remover or liquid brush cleaners.

Ball Point Pen: Blot immediately with a white cloth dampened in water or mineral oil.

Waxing and Refinishing: Improve wearability. Use any quality paste wax.

A diluted solution of 10% (or less) of bleach or clorox ay be used when cleaning this chair.

WARNING!


CASTERS MUST BE LOCKED EXCEPT DURING TRANSPORT

CASTER OPERATION

Note: This chair is equipped with four swivel locking casters for maneuverability.

Lock: To prevent the rear casters from and wheel turning, step on rear pedal of the caster until it lock.

Unlock: Step on the opposite end to unlock.

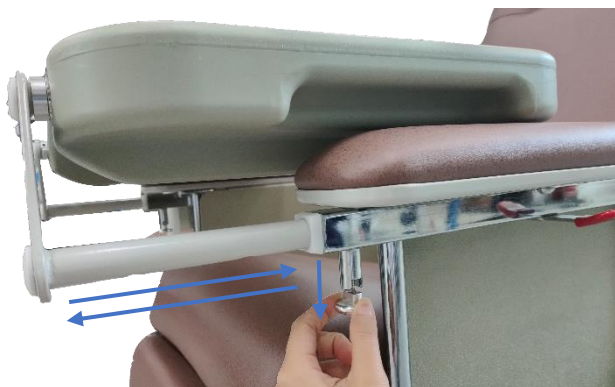

Slide Tables

To place either table in raised position: Raise table to the front and slide in until tabs engage slots in mount bracket. Pull / push the table to ensure lock tabs are engaged.

To return table to self-storing position: Pull the table outwards until tabs disengage slots.

Position / hook to slide of the side of the chair.



Articulating Headrest

Articulating headrest may be adjusted to a comfortable position by pulling forward or pushing back.

EXCESSIVE FORCE OR EFFORT IS NEVER NECESSARY TO CHANGE THE SEATING OF THE CHAIR. DO NOT OPERATE RECLINES BY APPLYING FORCE OR PRESSURE TO LEFREST



Seat Positioning

Reclining by attendant: Stand at either side of recliner. Grasp armrest with one hand and padded push handle with the other hand. To lower, push back on padded push handle. To raise, pull forward on padded push handle.

To return to seated position: To bring recliner back to the seated, no. 1 or no.2 position, lift the lever, slowly guide the back, up fully and release the lever. Return patient to desired position by pushing upward on padded push handle.



Recline Position 1



Recline Position 2



Lying Position