

# USER MANUAL – EURO ROLLATOR

## FOLDING PROCEDURE

### How to unfold:

1. Release the folding webbing Velcro strap.
2. Hold the handle and pull up until a click sound then secure with locking pin.
3. Push the seat side tubes down until a click sound.
4. Adjust the handle to the proper position.
5. Hook the bag on the bag holding rod and attach the Velcro to the frame.
6. Release the assisted brake to unlock the wheels.

### How to fold:

1. Push down the assisted brake to lock.
2. **Take off the shopping bag (better but not a must).**
3. **Pull up the folding webbing** : The folding webbing is positioned at the center of the seat. Pull it to fold the rollator horizontally to the Storage status.



4. Release the locking pin at the frame base.
5. Fold down the rollator frame.



6. **After folding, use the buckle webbing to fix the rollator:** Put the webbing on the rivet each side to secure the folded position



## SPECIFICATION

<b>Height:</b>	Max 36.6" Min: 31"	<b>Width:</b>	25" (unfolded) 10" (folded)
<b>Depth:</b>	23"	<b>Seat Width:</b>	18"
<b>Max. User Weight:</b>	136kg (300 lbs)	<b>Wheels:</b>	8"

## CLEANING METHOD

1. Wipe down your equipment weekly (or more often as needed) with a cloth dampened in a mild detergent/ water solution.
2. Allow the equipment to dry before using or storing.
3. Store your equipment safely away in a clean, dry location when not in use.

## WARNINGS

### FOR PERSONAL SAFETY

- For your health and comfort, be sure to use good posture and hold the rollator so that you can operate the brakes with ease.
- Please store the rollator in a safe environment.
- Unfold the rollator completely into the locked position before use.
- Possible risk of finger trapping - do not put fingers or hands into the hinge of the rollator when folding or opening for use.
- Ensure that the wheels are stable and on the ground before using.
- Ensure that the seat is locked in position before sitting.
- Always sit in the center of the seat, with the seat adjusted that the user has their feet on the floor.
- Always have the parking brakes locked in position before sitting.
- Never sit on the rollator when it is folded for storage.
- Never stand on the rollator, or use it as a step.
- Never transport a person on the seat of the rollator.
- Use caution when lifting and carrying the rollator.

## WARRANTY

There is a twelve-month warranty from the date of delivery of your new rollator. The warranty covers the frame only and against manufacturing defects; wear and tear parts not included.