

11. How to use wheelchair standing assist function

HR-WSR1000RJ / HR-WSR1100RK / HR-WSR1200RU

[When sitting]

1. Set the standing assist force to match the user's weight.
2. Fold the footrest of the wheelchair and make the wheelchair user's feet secure to the ground.
3. Place your hips on to the seat and pull the lever attached on the right side slowly.
4. Seating down is assisting the sitter to sit down .
5. When the seat is lowered to the desired position, release the lever to lock the seat position

[When standing]

1. Fold the footrest of the wheelchair and make the wheelchair user's feet secure to the ground.
2. While leaning forward, slowly pull the standing assist lever attached on the right side.
3. Seating up is assisting the sitter to stand up.
4. When the seat is raised to the desired position, release the lever to lock the seat position.

HR-WSR1000CJ / HR-WSR1100CK / HR-WSR1200CU

[When sitting]


1. Set the standing assist force to match the user's weight.
2. Fold the footrest of the wheelchair and make the wheelchair user's feet secure to the ground.
3. Place your hips inside the seat then the seat is goes down slowly.
4. You can sit down on the seat comfortably while the seat is going down.
5. When the seats come down, the seats are fixed.

[When standing]


1. Fold the footrest of the wheelchair and make the wheelchair user's feet secure to the ground.
2. The caretaker pulls forward the "pull switch" under the seat.
3. Hold the user's upper body and stand up.
4. It assists to stand up as the seat is raised.

12. How to lock up wheelchair standing assist device

HR-WSR1000RJ / HR-WSR1100RK / HR-WSR1200RU

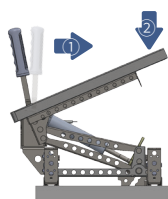


1. Put the standing assist device on the pad of storage.
2. By pulling the weight adjuster ring the weight setting position locates on 35kg.

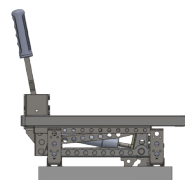


3. While pulling the lever(①), push it in the direction of the arrow on the seat(②).

4. When the seat is completely down, release the lever.

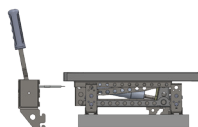


5. When the seat is securely folded, the knob of standing assist lever should be kept away from the main device.


HR-WSR1000CJ / HR-WSR1100CK / HR-WSR1200CU

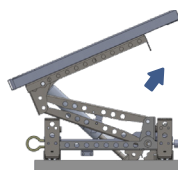

1. After detaching it from the wheelchair, put the standing assist device on the pad of storage.
2. Put the weight adjuster ring at the 35kg range location.

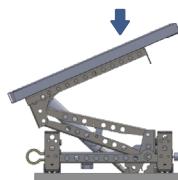

3. Press the part (↓) of the seat until it becomes locked.

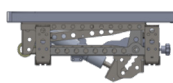

4. Keep it locked.

Wheelchair standing assist device

HiRize - WSR User manual



Warranty Certificate

Thank you for purchasing a product from Hize Co., Ltd
If a circumstance arises where you lose a receipt or the certificate of a guarantee, then the term of guarantee will be calculated 3 month (certified by the number of the manufacture, a certificate of inspection) after the date of manufacture. In case of accessories, only those that match the sales list manage by the service will be eligible for customer service

Hize Co., Ltd
Tel : +82-53-710-3323
E-mail : hizecoltd@gmail.com

Name of Product	WSR
Name of Model	HR-WSR1000RJ / HR-WSR1100RK HR-WSR1200RU / HR-WSR1000CJ HR-WSR1100CK / HR-WSR1200CU
Date of Purchase	
Manufacture Number	
Place of Purchase	
Term of Guarantee	1 year from purchase

Warranty

1. Hize Co., Ltd (hereinaft referred to as Hize) provides the warranty service for its products in compliance with the regulations on compensation for damage of customer.
2. Upon receiving a notification on a defect of its product within the range of warranty during the warranty period, Hize will repair or replace the defective product according to its warranty policy.
3. If the defective product within the range of warranty is not repairable or replaceable, Hize shall refund the customer. The replacement shall be equivalent to a new product in the aspect a notice of defect.
4. Hize assume no responsibility for repairing, replacement or refund until it receive the defect product from the customer. The replacement shall be equivalent to a new product in the aspect of performance.
5. Warranty is valid in the countries where the Hize products are distributed by Hize. (Contact for additional warranty service like field service)

1. To customers

Thank you very much for purchasing our revolutionary wheelchair standing assist device.

Be sure to read the Safety Precautions before use please. If you find any inconveniences or malfunctions during use, contact us please.

We are not responsible for accidents caused by misusing the product , not following the instructions of the user manual. Please keep this manual in a place where you can refer it to at all times.

2. In the box



User manual

Standing assist device


Pad for storage



Standing assist lever

Only for the following
3 products
HR-WSR1000RJ
HR-WSR1100RK
HR-WSR1200RU

3. Image of product

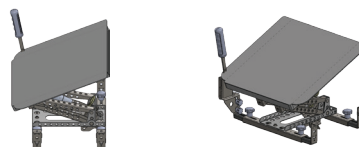

HR-WSR1000RJ / HR-WSR1100RK / HR-WSR1200RU




HR-WSR1000CJ / HR-WSR1100CK / HR-WSR1200CU

4. Safety precautions

- Be careful not to overuse each part and avoid from severe shock from outside, especially while operating a wheelchair.
- When using the standing assist function, be sure to put the wheelchair brake on a flat surface. If the brakes are not engaged, wobbly wheelchair movement might be dangerous.
- When using the standing assist function, be sure to fold the footrest and put your foot on the ground. Otherwise, the wheelchair may tilt forward and fall.
- The standing assist lever should be pulled over when the user is ready to stand up, tilting over his/her body slightly forward.
- When you install the wheelchair standing assist device on a wheelchair, be sure to hold your hand firmly wide open not to get injured. Otherwise, your hand may get caught between the wheelchair and the wheelchair standing assist device.
- Wheelchair standing assist device shouldn't be fallen apart with the wheelchair, when it is lifted up. Please use with precaution checking the wheelchair and standing assist device are properly coupled.
- Please adjust the width of the wheelchair standing assist device to adjust to the width of your wheelchair.
- If you are not using an standing assist device, Please keep it securely locked.
- First set up a weight adjustor suitable to your weight by using Weight adjustorrng and then pull the lever slowly. Your body can spring out of your seat if the weight adjustor is not properly adjusted.
- Do not load something on the seat when storing.

8. How to fit it to seat width

Wheelchair stand assist device (FELIZ) is designed to cope with a width of wheelchair.

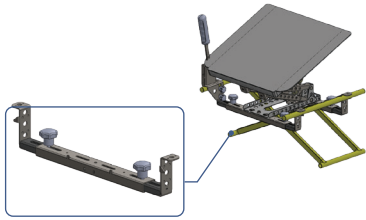


1. Loosen the adjustment knob of fixed wheelchair front/rear wing.
2. Holding the knobs by hand, adjust the width by pulling or pushing the fixed wheelchair front/rear wing. Align the width of the wheelchair and tighten the knobs to secure the width of the wheelchair standing assist device.

※ If you pull one side, the other side is supposed to move together also. (ex: If you are pulling 1cm on the right, the other side also moves 1cm on the left simultaneously and total distance would 2cm.)


9. How to attach it to a wheelchair

- After removing the seat cloth of the existing wheelchair, lift the wheelchair standing assist device and fix it into the location indicated below.
- Wheelchair standing assist device (FELIZ) can be stably mounted on the frame to which the wheelchair seat (cloth seat) is coupled.

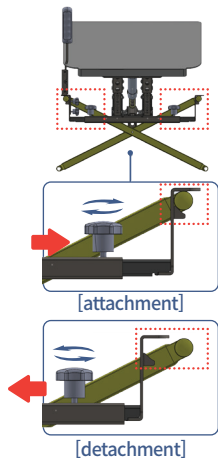

<Wheelchair coupling part –attachment>




- ① Unfold the wheelchair wing, insert it into the wheelchair seat connection frame as shown.
- ② Lock (turn clockwise) the adjustment knob of fixed wheelchair wing.
- ③ Use a Velcro to fasten wheelchair standing assist device and frame of wheelchair. (Refer to the figure)

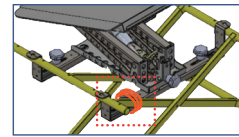

<Wheelchair coupling part – detachment>

- ① Use a Velcro to unfasten wheelchair standing assist device and frame of wheelchair. (Refer to the figure)
- ② Loosen (turn anticlockwise) the adjustment knob of fixed wheelchair wing.
- ③ Fold the fixed wheelchair wing to detach (separate) it from the wheelchair.

10. How to control weight adjustor ring (7step : 35~105kg)

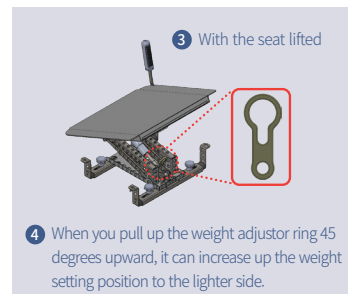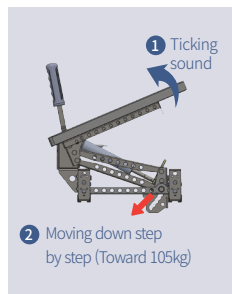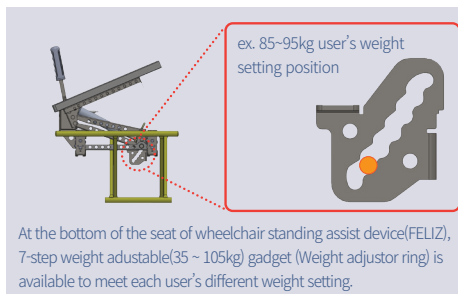

- Use the weight adjustor ring on the back of the wheelchair standing assist device to match the weight of the sitter.
- The larger the number, the stronger the standing assist force.
- You can use it to meet the desired standing assist force.

[How to use weight adjustor ring]

1. Place the seat of the wheelchair standing assist device in an upright position.
2. Putting the seat slightly up and down(ticking sound), the weight setting position goes down gradually from 35kg to 105kg.
3. If the weight setting position locates on 105kg position, raise the seat slightly lifting the weight adjustor ring at 45° upward to increase the weight setting position.

※ If you leave a weight adjustor ring at an unstable position, it can roll down automatically, so check the exact ticking sound and adjusted weight position.