
# b:joynz compression stockings

medizinische kompressionsstrümpfe  
compression médicale veineuse  
medias de compresión  
de uso médico

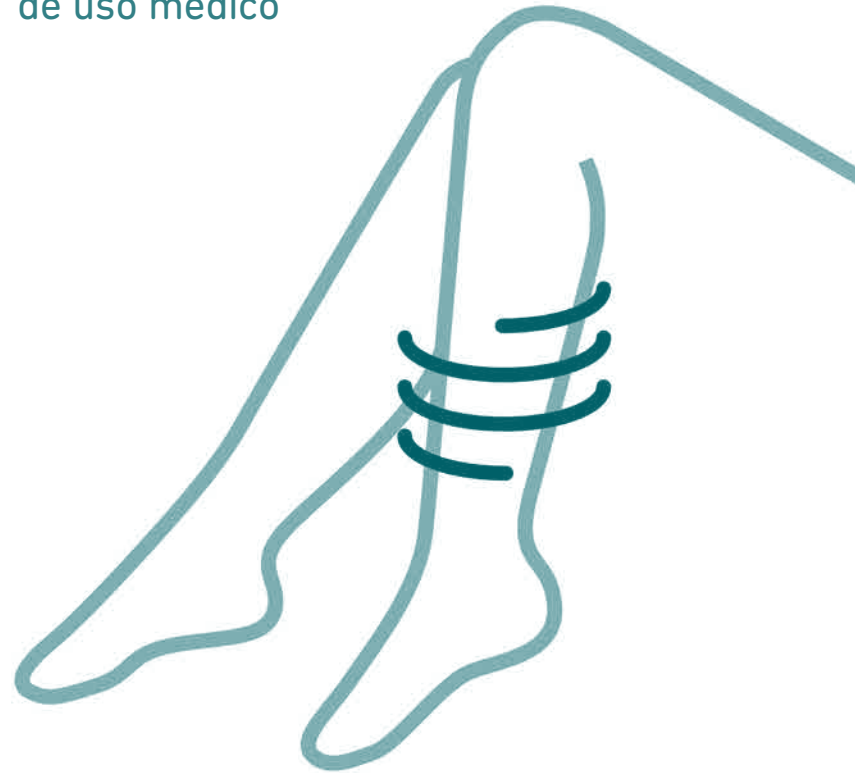be free  
be active

All instructions for use:  
<http://www.bjoynz.com/downloads>

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#### Intended purpose

bjoynz compression stockings are medical products. They are compression stockings for the long-term therapy of phlebological and / or lymphological disorders as well as, in special cases, for the long-term therapy of lipedema.

#### Indications

##### Venous symptoms

- Varicose veins / initial period following varicose vein therapy
- Prevention and treatment of venous edema
- Venous malformations
- Functional venous insufficiency (obesity, jobs involving sitting or standing)
- Improvement in quality of life for chronic venous disorders
- Prevention and treatment of venous skin changes
- Eczema and pigmentation
- Lipodermatosclerosis and atrophie blanche
- Prevention of recurring venous leg ulcers (CEAP stage 5)

##### Thromboembolism

- Superficial vein thrombosis
- Deep vein thrombosis
- Condition following thrombosis
- Post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients

##### Edema

- Phlebolympheoedem
  - Edema in pregnancy
  - Post-traumatic edema
  - Postoperative edema
  - Postoperative reperfusion edema
  - Idiopathic cyclical edema
  - Lipedemas
  - Congestion conditions due to immobility (arthrogenic congestive syndrome, partial or complete paralysis of the extremities)
  - Occupational edema (standing or seated occupations)
  - Drug-induced edema, if no adjustment is possible
- ##### Special case
- Lymphedema
  - Lipedema from stage II
  - Lipo-lymphedema
  - Scars

##### Other indications

- Obesity with functional venous insufficiency
- Inflammatory dermatosis of the leg
- Nausea and dizziness in pregnancy
- Congestion complaints in pregnancy
- Condition following burns

#### Part of the body this product is used for

In accordance with the indications

#### Risks of using this product

There are no known application risks if you use the product solely according to the indication and provided that you have been measured by a trained specialist and you have been taught the correct method for putting on the compression stockings. In the case of stockings with a silicone top band, skin irritation can occur in patients who are intolerant to this material.

Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

#### Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI < 0.5, arterial pressure at the ankle < 60 mmHg, toe pressure < 30 mmHg, or tcpO2 < 20 mmHg at the instep)
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the decision to treat should be made after weighing the benefits against the risks in order to choose the most appropriate means of compression:

- Dermatitis with significant weeping
  - Intolerance of the compression material
  - Severely impaired sensitivity of the extremity
  - Advanced peripheral neuropathy (e.g. in the case of diabetes mellitus)
  - Rheumatoid arthritis
- The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

#### Application instructions

- The initial fitting must only be performed by medically trained specialists.
- It is best to put on your stockings first thing in the morning, as soon as you get up, or after lymph drainage.
- Wear Bauerfeind gloves to put on and take off your bjoynz compression stockings compression stockings as this makes it easier to distribute the material evenly.
- Your legs must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off your jewelry before starting.
- Please avoid overstretching the stockings and do not pull the product up by the silicone top band.
- Avoid damaging the knitted fabric and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.

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- Store your bjoyzn compression stockings in a dry place away from direct light.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the label on the packet.

#### Cleaning instructions

- Please use mild detergent and a lingerie bag. Never expose your product to direct heat. Regular care guarantees optimum effectiveness of the product.
- The product can be machine dried provided a delicate cycle is chosen.
- When handwashing the stockings, we recommend drying them between two towels (do not wring).
- Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.
- Please wipe the silicone non-slip band on the inside once a day with a damp cloth.
- **Also observe the instructions on the sewn-in label on the upper edge of your product.**

#### Notes on reuse

This product is intended to be fitted to one patient only.

#### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the bjoyzn compression stockings have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

#### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

#### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

#### Material content

The following materials are used in our compression stockings: polyamide and elastane. For information regarding the composition in percent, please refer to the individual sewn-in label.

 – Medical device

 – DataMatrix Unique Device Identifier UDI

For more details, please consult your physician or medical retailer.

#### Tips for putting on the compression stockings Closed toe

- 1) Turn the stocking "inside-out" up to the heel. Pull the foot part over your toes and then over the instep.
- 2) Fold the leg part back over and pull it over the heel at the same time. Make sure that the heel is in the correct position.
- 3) Now gradually slide the stocking upward and distribute it evenly until there are no creases. Never pull from the top or the top band.

#### Open toe

- 1) Use the donning aid supplied.
- 2) Gather the leg part together and pull the foot part over the donning aid. Make sure that the heel is in the correct position.
- 3) Now gradually slide the stocking upward and distribute it evenly until there are no creases. Never pull from the top or the top band. You can now remove the donning aid by withdrawing it from the open toe.

**Please refer to the instructions provided on the first inside page of this brochure.**

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Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.