

# BIOME+™ MICROEXFOLIATION

## PROTOCOL



SKINCARE GOALS: **REJUVENATE, BALANCE & NOURISH**

**DESCRIPTION:** This skin-nurturing and microbiome-friendly treatment harnesses the power of microexfoliation (nanoneedling exfoliation technology) and enzymatic peel exfoliation to achieve superior results with IMAGE products. Features bt-titan® MN model AE with nano cartridges technology.

**SUGGESTED TREATMENT TIME:** 50 Minutes.

\*Prior to every treatment, review and have client initial and sign the IMAGE INFORMED TREATMENT CONSENT FORM

<b>01- PRE-CLEANSE</b>	Remove eye and lip makeup with <b>I BEAUTY REFRESHING FACIAL WIPES</b> . Apply <b>BIOME+ CLEANSING COMFORT BALM</b> and cleanse for 1-2 minutes. Remove with sponges and tepid water.
<b>02- CLEANSE</b>	Apply <b>ORMEDIC® BALANCING FACIAL CLEANSER</b> and cleanse with wet fingertips for 1-2 minutes. Remove with sponges and tepid water or aromatherapy steam towels.
<b>03- PREP</b>	Saturate rough gauze with <b>DEGREASING PREP SOLUTION</b> . Squeeze out excess solution. Using firm pressure, swipe gauze in both horizontal and vertical directions over the entire area to be treated.
<b>04- EXFOLIATE</b>	Combine 1 full dropper of <b>I ENHANCE HYALURONIC ACID FACIAL ENHANCER</b> with 1 full dropper of <b>BIOME+ DEW BRIGHT SERUM</b> in a medicine cup and apply to area of treatment with an <b>I SWAB</b> . Reapply as needed ensuring skin is kept moist. <ul style="list-style-type: none"> <li>• Divide the face into 5 sections (forehead, right side, left side, upper lip/chin, nose).</li> <li>• Work in 2-4" inch sections keeping skin taut. Use a vertical and horizontal pattern.</li> <li>• Repeat this pattern until the entire area of treatment has been covered.</li> </ul>
<b>05- LIFT</b>	Measure 1/8 oz. of <b>ORMEDIC® LIFT SOLUTION</b> into a medicine cup. Apply with an <b>I SWAB</b> in a circular motion beginning at forehead, moving to right cheek, left cheek, nose and perioral area. Massage for 1 minute. Solution may be left on the skin up to 5 minutes. Remove thoroughly with sponges and cool water until client feels ZERO activity.
<b>06- TARGET: BALANCE</b>	Apply <b>BIOME+ DEW BRIGHT SERUM</b> to entire treated area.
<b>07- EYE</b>	Apply <b>ORMEDIC® BALANCING EYE LIFT GEL</b> to orbital eye area.
<b>08- PREVENTION+</b>	Apply a generous amount of <b>PREVENTION+® DAILY TINTED MOISTURIZER SPF 30</b> to entire treated area.
<b>09- LIP</b>	Apply <b>ORMEDIC® BALANCING LIP ENHANCEMENT COMPLEX</b> to entire lip area.
<b>10- YANA</b>	Offer client a complimentary sample of <b>YANA® DAILY COLLAGEN SHOTS</b> .
<b>11- POST TREATMENT</b>	Provide <b>IMAGE POST TREATMENT KIT</b> to client and review post treatment care.