

IMAGE SKINCARE & BIO-THERAPEUTIC MICROEXFOLIATION PROTOCOL WITH ORMEDIC LIFT



CLEANSE Remove eye and lip makeup with **I BEAUTY REFRESHING FACIAL WIPES**. Apply **AGELESS TOTAL FACIAL CLEANSER** and cleanse with wet fingertips for 1-2 minutes. Remove with sponges and tepid water or aromatherapy steam towels.

PREP Saturate rough gauze with **DEGREASING PREP SOLUTION**. Squeeze out excess solution. Using firm pressure, swipe gauze in both horizontal and vertical directions over the entire area to be treated.

EXFOLIATE Combine 1 full dropper of **I ENHANCE HYALURONIC ACID FACIAL ENHANCER** with 1 full dropper of **I ENHANCE ANTIOXIDANT FACIAL ENHANCER** in a medicine cup and apply to area of treatment with an **I SWAB**. Reapply as needed ensuring skin is kept moist.

- Divide the face into 5 sections (forehead, right side, left side, upperlip/chin, nose).
- Keeping skin taut, perform 2-4 passes with the bt-titan MN, or until desired level of exfoliation has been achieved. Work in a vertical and horizontal pattern. Repeat this pattern until the entire area of treatment has been covered.

LIFT Measure 1/8 oz. of **ORMEDIC LIFT SOLUTION** into a medicine cup. Apply with an **I SWAB** in a circular motion beginning at forehead, moving to right cheek, left cheek, nose and perioral area. Massage for 1 minute. Solution may be left on the skin up to 5 minutes. Remove thoroughly with sponges and cool water until client feels ZERO activity.

TARGET-BALANCE Apply **ORMEDIC BALANCING ANTIOXIDANT SERUM** to entire treated area.

EYE Apply **the MAX™ EYE CRÈME** to orbital area.

PROTECT Apply a generous amount of **PREVENTION+ DAILY HYDRATING MOISTURIZER SPF 30** to entire treated area.

FLAWLESS FINISH Apply **I CONCEAL** using **FOUNDATION BRUSH** to create a flawless finish.

PLUMP Apply **ORMEDIC BALANCING LIP ENHANCEMENT COMPLEX** to entire lip area.

YANA Offer client a complimentary sample of **YANA DAILY COLLAGEN SHOTS**.

POST TREATMENT Provide **IMAGE POST TREATMENT KIT** to client and review post treatment care.

*Device used: bt-titan® MN (model AE with nano cartridges)