

MICROCURRENT PROTOCOL

SKINCARE GOALS: **HYDRATE AND LIFT**



DESCRIPTION: Perfect for all skin types, this protocol uses microcurrent technology with IMAGE formulas that leave skin with a radiant, refreshed appearance. Features bt-nano technology.

TREATMENT TIME: 50 Minutes

01- TREATMENT PREP	Prep technology with clean cotton buds or stainless-steel tips. Fill a small bowl with 1/2 ounce of Polarized Water. Dip microcurrent probes in product regularly during application ensuring consistently wet application.
02- PRE-CLEANSE	Remove eye and lip makeup with I BEAUTY REFRESHING FACIAL WIPES . Apply ILUMA® INTENSE BRIGHTENING EXFOLIATING CLEANSER and cleanse with wet fingertips for 1-2 minutes. Remove with sponges and tepid water or aromatherapy steam towels.
03- CLEANSE	Apply the MAX™ CLEANSER and cleanse with wet fingertips for 1-2 minutes. Remove with sponges and tepid water or aromatherapy steam towels.
04- EXFOLIATE	Dispense MICRODERM EXFOLIATOR into a medicine cup or flexi-bowl. Apply with an I SWAB in a circular motion to entire area of treatment. Massage gently for 1-3 minutes. Remove with dry gauze followed by sponges and tepid water or aromatherapy steam towels.
05- TARGET: HYDRATE AND LIFT	Combine 1 full dropper of I ENHANCE HYALURONIC ACID FACIAL ENHANCER with ORMEDIC® BALANCING GEL MASQUE in a medicine cup and apply with an I SWAB to the area of treatment. Keep a small bowl of Polarized Water to dip cotton buds in throughout treatment: <ul style="list-style-type: none"> • Select Invisible Expression Mode, Program 1 on the bt-nano. • Perform lifting and holding techniques according to technology program and manual. Remove masque with sponges and tepid water or aromatherapy steam towels.
06- BRIGHTEN	Combine 1 full dropper of I ENHANCE ANTIOXIDANT FACIAL ENHANCER with ILUMA INTENSE BRIGHTENING SERUM and apply to entire treated area.
07- NOURISH	Combine 1 full dropper of the I ENHANCE HYALURONIC ACID FACIAL ENHANCER with 1 pump of ORMEDIC BALANCING ANTIOXIDANT SERUM in a medicine cup and apply to entire treated area. Keep remaining product aside to dip cotton buds in: <ul style="list-style-type: none"> • Select Invisible Expression Mode, Program 2 on the bt-nano. • Perform skinwork techniques according to technology program and manual. Massage any remaining product into the skin.
08- SMOOTH	Dispense the MAX WRINKLE SMOOTHER into a medicine cup and apply with an I SWAB to forehead and around orbital area. Perform light tapotement.
09- CONTOUR	Massage the MAX CONTOUR GEL CRÈME into lower portion of face, including cheeks, jawline and perioral area. Apply in upward strokes.
10- EYE	Apply ILUMA INTENSE BRIGHTENING EYE CRÈME to orbital area.
11- FINISHING	<ul style="list-style-type: none"> • Select Invisible Expression Mode, Program 3 on the bt-nano. • Perform skinwork techniques according to technology program and manual.
12- PREVENTION+	Apply a generous amount of PREVENTION+® DAILY ULTIMATE PROTECTION MOISTURIZER SPF 50 to entire treated area.
13- I CONCEAL	Apply I CONCEAL using FOUNDATION BRUSH to create a flawless finish.
14- LIP	Apply ORMEDIC BALANCING LIP ENHANCEMENT COMPLEX to entire lip area.
15- YANA	Offer client a complimentary sample of YANA® DAILY COLLAGEN SHOTS .
16- POST TREATMENT	Provide IMAGE POST TREATMENT KIT to client and review post treatment care.