

INQLINE® ASSIST




IMPORTANT SAFEGUARDS AND WARNINGS

- This is a **quick guide** which **MUST** first be read and understood before using this system.
- **J-Hooks** must be attached to a solid wheelchair frame (no spokes, wheels or movable components).
- This is **NOT** a securement system and should not be used during transportation.

BOARD WHEELCHAIR PASSENGER

1


1. Wheelchair behind ramp.
2. Lock brakes or power off chair.
3. Press and hold both buttons (4 sec) to release retractors.

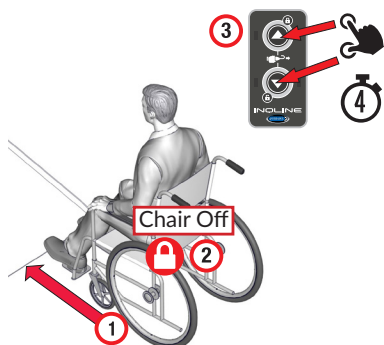

DEBOARD WHEELCHAIR PASSENGER


1

1. Wheelchair facing INQLINE Loader retractors.
2. Lock Brakes or power off chair.
3. Press and hold buttons (4 sec) to release retractors.
4. Attach front J-Hooks to solid frame (60 sec) and unlock chair.

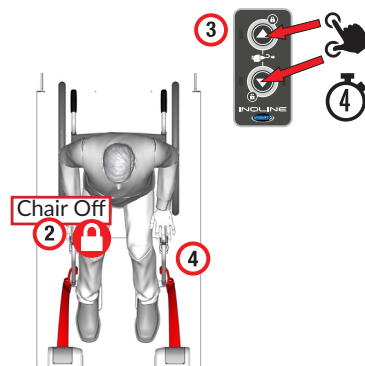


2

4. Attach front J-Hooks to solid frame (60 sec).
5. Unlock brakes and keep chair powered off.
6. Press and hold Z Button.
7. Push Thumb Controller forward—moves chair up ramp.
8. **CAUTION:** Move Controller left or right to adjust chair's direction.
9. Release Z Button and place Controller in storage.

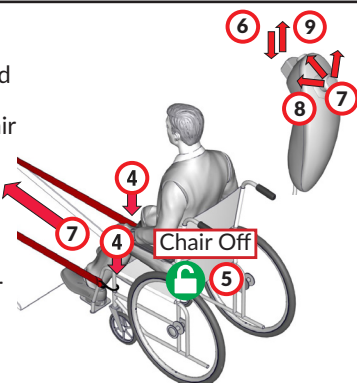


2

5. Press and hold Z Button.
6. Push Thumb Controller downward; moves chair down ramp.
7. **CAUTION:** Move Controller left or right to adjust chair's direction.
8. Release Z Button and place Controller in storage.
9. Lock brakes and keep chair powered off.

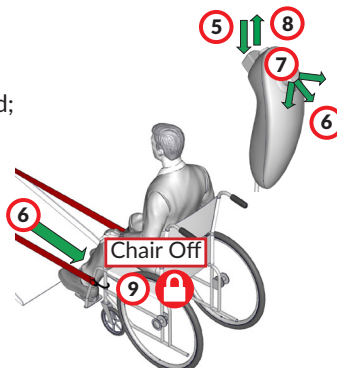


3

10. Wheelchair facing INQLINE Assist retractors.
11. Lock Brakes or power off chair.
12. Press and hold buttons (4 sec) to release retractors.
13. Detach J-hooks and return them to Stowage.
14. Unlock chair and move to proper securement area.

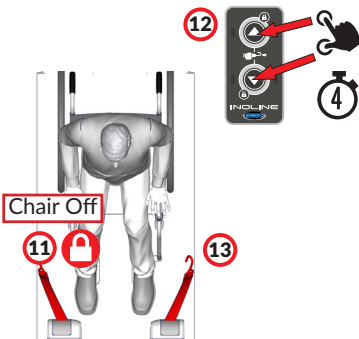

3

10. Press and hold both buttons (4 sec) to release retractors.
11. Return J-Hooks to stowage.
12. Unlock brakes or power on chair.
13. Place chair in secure area.

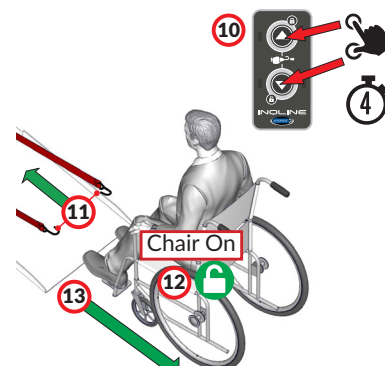

❓ Contact a Q'Straint® Authorized Dealer or Customer Service (800) 987-9987
www.qstraint.com

These instructions contain the most current information at the time of printing. Q'Straint® reserves the right to alter or modify contents and/or components without notice. © 2019 Q'Straint®. All Rights Reserved.