

Installation Instructions for the W008, W010 & W011 Cane and Reacher Holders

Conveniently mount snap in cane and extended reacher
holders to walkers and rollators.



**Select appropriate
clamp for the tube
diameter and place it
around the tube in
location cane or
reacher holder is to be
mounted.**



**Hand tighten supplied
screws with #2 Philips
screwdriver until unit
is secure.**

Do not over tighten!

Note: The three clamps are designed to fit a wide
variety of tube diameters and will not clamp
snuggly to some tube sizes until tightened with the
screwdriver.