


GLUCO III User's Manual

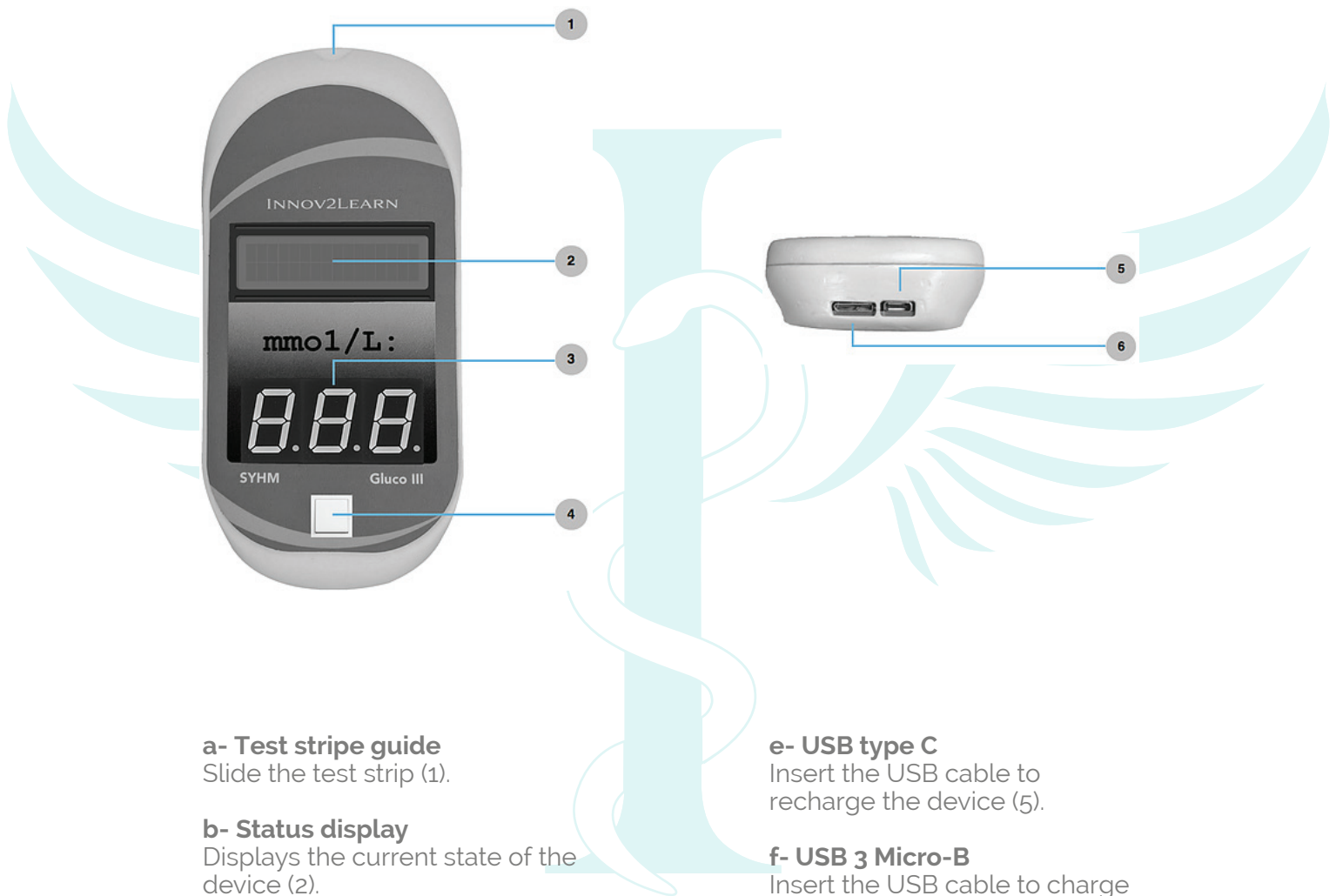

a- Test stripe guide

Slide the test strip (1).

b- Status display

Displays the current state of the device (2).

c- Result display

Displays the current test results (3).

d- ON / OFF button

Turns the device ON or OFF, and displays current battery status.

- Press and hold the button for 3 seconds to turn the device ON or OFF.

e- USB type C

Insert the USB cable to recharge the device (5).

f- USB 3 Micro-B

Insert the USB cable to charge the device (6).

GLUCO III User's Manual

How it works

1- Turn on the device

Press and hold the On / Off button for about 3 seconds.

2- Check the device status

The status display is now ON and Ready.

3- Connect to device

Open the SYHM app.

3.1 - Android/ iOS apps:

- Open the app, select the language and then click OK.
- Click on Glucometer.
- Your device must be shown under Available Devices.
- Click on the device name.

3.2 - Windows / Mac OS X apps:

- Connect the USB bluetooth dongle to your PC.
- Open the app and click on "Add Device +".
- Your device must now appear on the list.
- Click on the device name.

4- Insert the test strip

Slide the test strip into the test strip guide.

5- Send the test results to the device

The application is now showing a notification indicating that the device is ready to receive the results.

Enter the desired value and click on the "Send" button.

6- Check the test results

Test results appear on the Results display of the device.

Recharging the Battery

When the battery is low, the ON/OFF button starts flashing with the following pattern: two quick flashes followed by one long flash.

To recharge the device, plug the charge in one of the USB ports. Both can be used to recharge the device. Which one to use is up to you.

While the battery recharges, the ON/OFF button will flash constantly.

When the recharge is finished, the light on the ON /OFF button will remain ON, indicating that the battery is 100%.