
ADROIT[®]

SURGICAL

VOIR BOUGIE[®]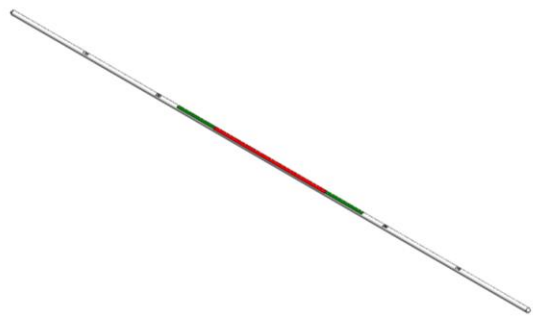

Truer Medical, Inc.

AN ISO 13485
REGISTERED COMPANY

VOIR BOUGIE[®] Endotracheal tube
introducer with Safety Markers
5mm x 70cm For use with endotracheal tubes 6.0mm ID
and larger. The Bougie is for adult single use. Do not
reprocess.

DFU – 050 Rev C (Oct 19)

Indications for Use:

The Voir Bougie is designed to help guide the direction of the endotracheal tube (ETT) between the vocal cords. The Voir Bougie has clearly marked colored bands that allow the user to quickly insert the Bougie the correct distance.

Instructions for Use:

- Bougie may be lubricated with water-soluble lubricant or used dry.
- When using a regular laryngoscope and the glottis is not easily visualized carefully insert the Bougie, coude tip first, below the epiglottis in the direction of the vocal cords. Other laryngeal landmarks such as the arytenoids may help the user pass the Bougie towards the glottis. Advance the Bougie into the trachea. Clicks may be felt as the Bougie passes against tracheal rings. Stop advancement of the Bougie once the 'green' safety band has just passed the upper teeth/gum. **Do not insert the Voir Bougie beyond the first green band (red band is an absolute insertion stop point).**
- It is recommended the user pass either end of the Bougie through the vocal cords using a Vie Scope. Stop advancing the Bougie once the green band just passes the upper teeth/gum. Remove the Vie Scope leaving the Bougie in the same position as noted by the colored band.
- Pass the ETT over the Bougie using standard methods. Stabilize the Bougie as the ETT passes over to prevent further intrusion into the trachea. The colored band facilitates keeping the Bougie in the correct position.
- Stabilize the ETT as the Bougie is removed to prevent dislodgement and verify proper ETT placement.
- Discard Bougie in accordance with standard hospital protocol.

Contraindications: There are no absolute contraindications, however, the user must exercise caution when using the Bougie in traumatic situations where visualization is difficult to avoid perforation of the pharynx. The user should feel little or no resistance when inserting the Bougie.

Precautions: If you encounter resistance, do not force the Bougie; may cause vocal cord injury, soft tissue trauma, bronchial rupture, or pneumothorax.

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