

RODIN™ BOND

Dental Adhesive System

THE FIRST UNIVERSAL
BONDING AGENT


DESIGNED FOR

3D PRINTED
RESTORATIONS



ALSO DELIVERS

- *Highest Shear Bond Strength for conventional zirconia and composite restorations*
- *Maximum adhesive strength when coupled with SensiGuard desensitizer, ensuring patient comfort*

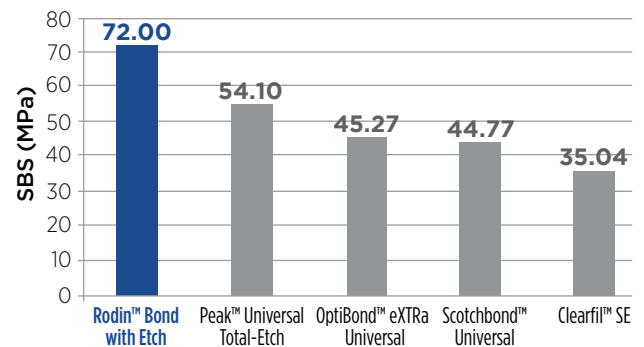

*Data from an Independent Research Institute**

RODIN™ BOND Dental Adhesive System



“Rodin Bond demonstrates excellent performance even under high-impact and challenging posterior restorations, with patients who previously experienced post-operative sensitivity reporting no discomfort.

-Dr. Raihan Nazir, DDS, MSc (FFDRCSI)



Product Features


- Unique nanohybrid matrix technology provides exceptional bond strength for 3d printed restorative applications
- Highest adhesion strength for conventional restoratives including Zirconia and composite restorations*
- Efficacy on direct and indirect restorations is further enhanced by SensiGuard™, a dual-function primer and desensitizer
- Flip-top cap design provides “one-drop” flow of adhesive while innovative Unidose system reduces exposure to air and moisture, preserving the bond agent’s effectiveness

*As verified by an independent research institute.

Multiple Delivery Options



Bottle: economic design, precise drop dispensing eliminates waste

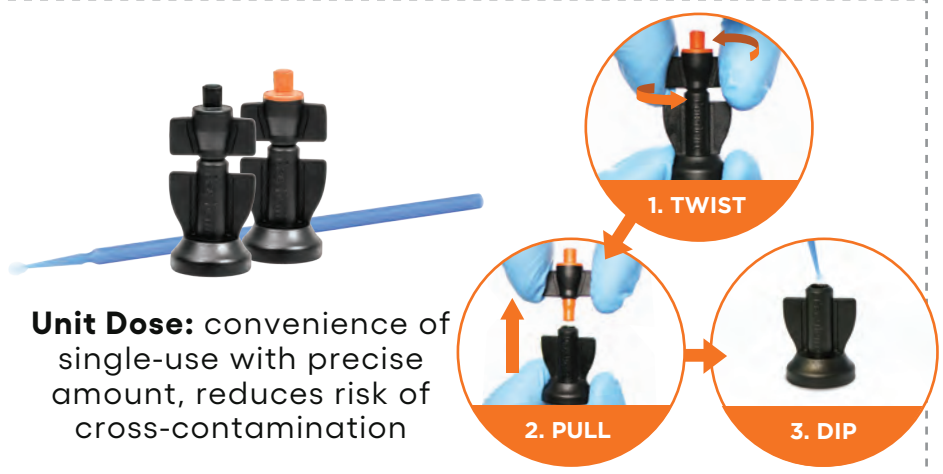

Unit Dose: convenience of single-use with precise amount, reduces risk of cross-contamination



RODIN™ BOND Dental Adhesive System **WORKFLOW**



STEP 1
Etch enamel and dentin with **PacEtch™**



STEP 2
Apply **SensiGuard™** 2-in-1 desensitizer and primer



STEP 3
Apply **Rodin™ Bond Adhesive**




STEP 4
Place **Rodin™ Sculpture** crown. Light cure surface.



SENSI GUARD™

When combined with a dental adhesive such as Rodin™ Bond, SensiGuard™ acts as both a desensitizer and a primer. Its unique formulation revitalizes collapsed collagen fibers, providing unparalleled monomer penetration, creating exceptional shear bond strength and steadfast adhesion to dentin tubules.

- **Patient Comfort:** Significant reduction of post-operation sensitivity in patients
- **Anti-Bacterial:** Potent antibacterial qualities reduce caries formation beneath restorations
- **Visibility:** Distinct yellow tint provides unparalleled visibility during procedures
- **Universal Compatibility:** Seamless integration with a wide range of dental adhesives and workflows

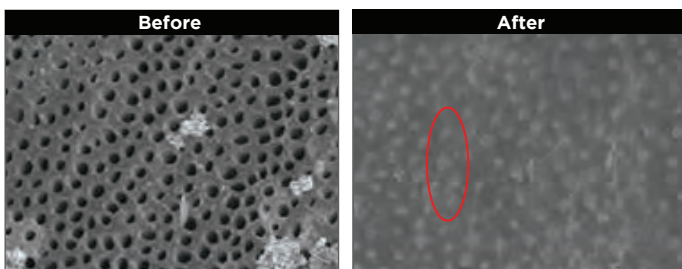

Longitudinal slices of a sample, subjected to SensiGuard™ treatment, were examined at magnifications of x5,000. The circled region indicates an obstructed area*.

*Data on file.



Scan QR Code to Visit Rodin-3D.com to Learn More about Rodin 3D Resins for Printed Restorations.



RODINTM BOND Dental Adhesive System

	Item#	Product Description
	471101	Rodin Bond Dental Adhesive & SensiGuard Desensitizer Intro Kit 1 x 5 mL Rodin Bond Bottle 1 x 5 mL SensiGuard Bottle 2 x 1.2 mL 38% PacEtch Pro Syringe 100 x Micro Brush regular brush 20 x Mixing Wells, 4 holes 20 x Dispensing tips for PacEtch Pro 1 x Technique Guide
	481102	Rodin Bond Dental Adhesive System Unit Dose Kit 50 x Rodin Bond 0.1 mL Unit Dose 50 x SensiGuard 0.1 mL Unit Dose 1 x Technique Guide 1 x Instruction Booklet
	471102	Rodin Bond Dental Adhesive Refill, 5mL Bottle
	471103	Rodin Bond Dental Adhesive Unit Dose Refill, 100/pk
	481101	SensiGuard Desensitizer Refill, 5ml Bottle
	481103	SensiGuard Desensitizer Unit Dose Refill, 100/pk
	135 135E 135C 231	PacEtch Pro 38% Phosphoric Acid Etch 4 x 1.2 mL syringes, 8 x Dispensing Tips Econo pack: 36 x 1.2 mL syringes Clinic pack: 144 x 1.2 mL syringes 25 Ga tips, Etch, Blue, 100/pk
	901 902 903 904	Disposable MicroPro Applicators Standard 2.0mm, 400/pk Disposable MicroPro Applicators Fine 1.5mm, 400/pk Disposable MicroPro Applicators Ultra fine 1.0mm, 400/pk Disposable MicroPro Applicators Tapered, 400/pk
	906	Mixing Well, 4-Well, 200/pk