


ENDO-AID INSTRUCTIONS FOR USE

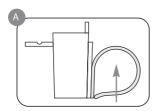

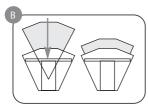

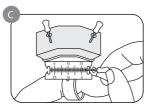

- A Before assembly, choose one of two available ring sizes (large and small) that best fits your index finger. Slide the selected ring into the cup channel located beneath the scale.
- B Place the foam insert into the cup. Push the foam insert all the way into the bottom of the cup to seat the foam.
- C Arrange hand files, rotary files or other instruments into the foam insert, Remove all file bioburden prior to autoclaving files in the foam insert. When using rubber stops, utilize the stop locks on either the built-in ruler or the included ruler to increase or decrease the instrument's working length. The rulers perform well with paper points, gutta percha and many other endodontic instruments. To efficiently clean instruments simply pass them in and out of the foam insert. You can also clean instruments by sliding them through the corrugated area on top of the foam insert. Rubber stops can be safely added to instruments using the ports located on the back of the cup.

	ULTRASON I C PRE CLEAN	STEAM AUTOCLAVE	CHEMI- CLAVE	ETHYLENE OX I DE	DRY HEAT	COLD STERILE	
Ring Organizer	YES	YES	YES	YES	NO	YES	



Warning: Do not place any components into Surgical Milk or instrument dips prior to autoclaving.



