

ARMOR™ PSP Barrier Envelopes



Intended Use: Armor™ PSP Barrier Envelopes are intended to be used as a disposable barrier for Phosphor Storage Plates. This device is non-sterile and intended for single patient use only.

Instructions:


1. Insert the Phosphor Storage Plate



2. Peel off the white adhesive strip



3. Seal the Armor™ PSP Barrier Envelope

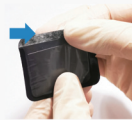

4. Tear open the Armor™ PSP Barrier to remove the plate after use.



Important:

Dispose the Armor™ PSP Barrier Envelope in accordance with all local regulations.

Do not use an Armor™ PSP Barrier Envelope that has been punctured or torn.

Armor™ PSP Barrier Envelope Ordering Information

PPE-732-0	Size 0 side loading barrier envelope, 100/box
PPE-732-1	Size 1 side loading barrier envelope, 100/box
PPE-732-2	Size 2 side loading barrier envelope, 100/box
PPE-732-3	Size 3 side loading barrier envelope, 100/box
PPE-732-4	Size 4 side loading barrier envelope, 50/box



**Manufactured for
Pac-Dent, Inc.**

670 Endeavor Circle
Brea, CA 92821 USA
909.839.0888
Made in China Rev.091523

