

[PRODUCT NAME] : Isolation clothing

[MODEL SPECIFICATION] : S、M、L、XL、XXL、XXXL

[PRODUCT PERFORMANCE]

The company declares the fracture strength of the material in the key part of the product should be no less than 40N in the longitudinal direction and no less than 25N in the transverse direction. Elongation at break shall not be less than 15%

[MAIN STRUCTURAL COMPOSITION]

Using non-woven as the main raw materials, through cutting, sewing made. Non-sterile and disposable

[APPLICATION]

Used for general isolation in clinic, ward, laboratory, etc

[CONTRAINDICATION] : Temporarily not found

[PRODUCTION DATE AND SHELF LIFE]

PRODUCTION DATE: See The Label

SHELF LIFE: 3 years

[NOTES, WARNINGS AND REMINDERS]

- 1.set/bag of this product , Please make sure the package is in good condition before use and read this manual carefully
- 2.This product is for disposable use only. Please make sure the zipper is closed before wearing this isolation suit to enter the work site.
- 3.If the clothing is contaminated, worn or damaged, it should be stopped immediately and disposed of properly
- 4.Please confirm your working environment before use. Our company will not be responsible for any damage caused by exceeding the use or limit of this product.
- 5.Please make your own decision whether to wear other protective equipment according to your specific situation
- 6.Please clean and disinfect the hands before wearing and after disconnecting
- 7.This product is flammable, please keep away from heat source and open flame when using or storing
- 8.Please avoid direct sunlight and storing in damp environment

[APPLICATION METHOD]

1、PUT ON ISOLATION CLOTHING:

Open the Isolation clothes , check if it damaged , pull the zipper→The left and right hand holds the isolation clothing one-piece cap and sleeve,grab the zipper opening at the waist of the isolation clothing to avoid contacting with the ground→Dress on the bottom first,then the top , after that zip it to the chest, then fasten the cap to the head and zip it.


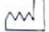




2、TAKE OFF THE ISOLATION CLOTHING:

Zip down to the bottom→Pull the cap up so that the head is out of the cap,Take off the sleeves→Roll from top to bottom→take off the clothes, Turn the pollution inward and place it in a clinical waste bag.

[STORAGE] : The product should be stored in a dry, ventilated room away from direct sunlight, Storing 10cm above the ground , 20cm away from the wall; The limit of stacking layers is 6

[PACKAGE]

- 1.Minimum packing unit: 1 set/bag
- 2.For special packaging, the form of packaging shall be determined according to customer requirements, and the quantity shall be recorded on the Label

Graphic Symbol	Symbol Description	Graphic Symbol	Symbol Description	Graphic Symbol	Symbol Description
	Do Not Reuse		Date Of Manufacture		Manufacturer
	Batch Code		Use By		Authorised representative in the european community



CHENGDU SANI MEDICAL EQUIPMENT CO.,LTD

Building4,No.2401 West Port Road,Chengdu,
Sichuan Province,610299,P.R.China

Phone: 0086-28-83361652 Fax: 0086-28-83361651



SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

