



# Quick Start Guide – Installation & Setup


Installing




1. Rings




2. Short tube

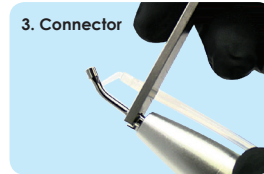


3. Connector

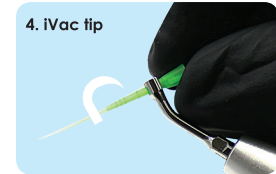


4. iVac tip

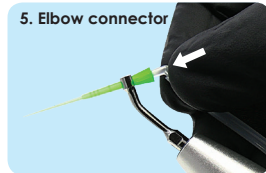

5. Elbow connector

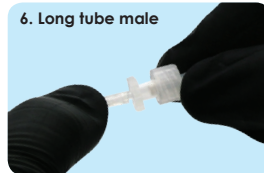

6. Long tube male

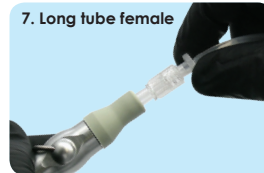


7. Long tube female



8. Capillary .60

## Concomitant irrigation from the reservoir or bottles

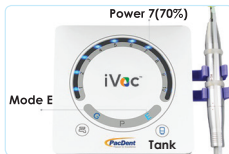

1. Power 70%. Mode E. Reservoir icon



2. Bottle



3. Irrigation volume adjustment

## Concomitant irrigation with syringe and cannula



1. Power 70%. Mode E. Water line icon



2. Syringe and Cannula .60



- Press twice E button to activate the 30sec timer
- Dripping irrigation fluid at pulp chamber only
- Only activate the iVac tip inside the canal
- Flush with distilled water after use (Flush Mode)

Consult the IFU for more information







Scan QR Code  
for more information.





# Quick Start Guide – Final Irrigation Protocol Suggestion

## Concomitant irrigation from the reservoir or bottles

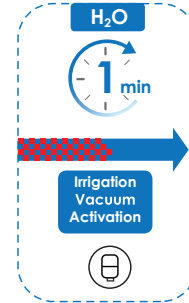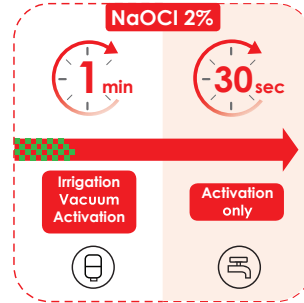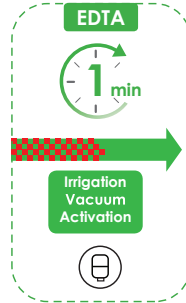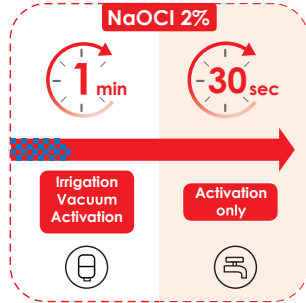

## Concomitant irrigation with syringe and cannula

