Technische Information  
*Technical Information*

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## **Silicone Vacuum Sealing Ring**

### **1. Application**

*The LITE Vacuum Ring 5W700 is available in 4 sizes. It is suitable only for use in prostheses of the lower and upper extremities. The use of the 5W700 is intended for use with almost all fabric-coated liners from silicone, polyurethane or TPE.*

### **2. Description**

*The LITE Vacuum Ring 5W700 can be glued in any position with almost all fabric-coated liners. Use the 7W631 adhesive silicone.*

*The LITE Vacuum Ring 5W700 can be used with active or passive vacuum system. The LITE Vacuum Ring 5W700 can be used together with Locking Liners to prevent rotation and longitudinal pistoning.*

*The advantage of the LITE Vacuum Ring 5W700 is that a vacuum is created between the socket and the liner so that the prosthesis is held against the body without the need for a knee cap or locking device.*

### **3. Technical Data**

Art. No.	Material	Circumference
5W700-2	Silicone	16 – 22 cm
5W700-3	Silicone	20 – 38 cm
5W700-4	Silicone	30 – 48 cm
5W700-5	Silicone	40 – 55 cm

#### **4. Handling**

##### **4.1 Usage and gluing of the LITE Vacuum Ring 5W700**

*Determine location of the ring on the liner. Use the plaster cast therefor. In case there is already a valve in the prosthesis socket, it should be noted that the vacuum ring is posterior to the valve seat and this is not affected. Mark this through the valve hole in the prosthesis socket to position the seat of the valve. Glue above and below the ring with masking tape 7W427-30. A gap of about 3-5 mm should remain. Remove the ring from the liner and clean with 7W910 acetone on the side to be bonded. Apply 7W631 adhesive silicone to the dry, greasy and lint-free fabric between the 7W427-30 masking tape. Make sure the seams are coated well. Position the ring on the silicone adhesive and wipe off excess silicone. Make sure that no air is trapped between the ring and the adhesive. These cause air pockets and reduce the life time of the silicone ring. Remove the tape and smoothing bonding silicone on the edge of the ring. Let the bonding silicone at least 4-6 hours to cure (preferably overnight). When using the LITE Vacuum Ring 5W700 the use of a 4W294 liquid donning aid is recommended which is sprayed before donning on the ring. This evaporates quickly and does not limit the function of the vacuum LITE Ring 5W700.*

#### **5. Risks**

*If the recommendations given have been observed, no special risk is to be expected when using the 5W700 LITE Vacuum-Ring. The functioning of the 5W700 LITE Vacuum-Ring is to be checked in regular intervals but at least every 6 months.*

#### **6. Warranty**

*This warranty applies to faults in material and workmanship and is granted to orthopedic workshops only if all given proceeding instructions are complied with and the products are used if intended. The functioning of the adaptor is to be checked in regular intervals but at least every 6 months. The product has been tested according to ISO recommendations. Wagner Polymertechnik offers a warranty of 3 months in case of appropriate use.*

**7. Declaration of Conformity**

*The product meets the requirements of guideline 93/42/EEC for medical products. Based on the classification criteria for medical products according to Appendix IX of the guideline, the product was classified as Class I. Therefore, the conformity declaration was prepared under the sole responsibility of Wagner Polymertechnik GmbH according to Appendix VII of the guideline*

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Wagner Polymertechnik GmbH has been certified by the TÜV