

## QUICK REFERENCE GUIDE

FEATURE BATTERY/POWER STATUS BUT		STATE	BEHAVIOR
BATTERY/POWER STATUS BU	TTON		
	TION		
	PRESS & RELEASE	Battery Fully Charged	Solid green (51-100%)
Check Battery Status		Battery Charged	Blinking green (21-50%)
		Battery Low	Solid red (11-20%)
		Battery Very Low	Blinking red (1-10%)
Reboot Knee	HOLD 6 SEC	*Only do this if you are having problems with your Knee and directed to by your prosthetist	Blinking green and blue
MODE BUTTON			
Check Activity	PRESS & RELEASE	Unlocked	Walking: Slow blinking white Alternate: Fast blinking white
		Locked	Error: Blinking red/white No Activity is active to check because the Knee is Locked
Enable/Disable Flexion Lock	DOUBLE PRESS	Locked	Lock Engaged: Solid white Locks further flexion, but allows extension
		Locked with Very Low Battery	Lock Engaged: Solid white / Battery Button blinking <mark>red</mark> Locks further flexion, but allows extension
		Unlocked	Lock Disengaged: No light Previously selected Activity resumed
Toggle between Walking and Alternate	LONG PRESS	Current Activity: Walking	Slow blinking white
		Current Activity: Alternate	Fast blinking white
		Current Activity: Walking Alternate Activity has not been enabled in the patient app or it is currently unsafe to switch Activities (i.e., Knee is bent)	Blinking red/white
WIRELESS BUTTON			
Pair Knee with App	PRESS	Pairing Devices	None
STORAGE MODE *Do not pu	it into Storage Mo	ode if no charging source is available to wake the	e Knee up
Enter Storage Mode *Do not plug into a power source	HOLD TOGETHER 6 SEC	Power Down *Knee will not enter Storage Mode if connected to a power source	Blinking red / blue / white
EXIL Storage Mode	Plug into power source	Power On *Must plug into power source to exit Storage Mode	Battery Button: Solid green / Wireless Button: Blinking blue *LEDs will turn off after fully exiting Storage Mode
LED REFERENCE			
System Error			Alternate red / blue
Booting			Blinking green / blue

