

Care Instructions HealthCare Mattresses

IMPORTANT: Do not clean with abrasive cleaners, bleach or by washing machine. Adhere to Manufacturers recommendations on usage and dwell times. A good cleaning regime, rinsing and drying after disinfection, will significantly enhance the life of the mattress cover.

5 Steps to Cleaning / Disinfecting Mattress Covers: Use RTU (**Ready To Use**) cleaners / disinfectants (see recommended products below)

1. Remove all contamination from surface i.e. urine, feces, blood with absorbent material i.e. cloth, paper towels etc.
2. Ensure area is clean by wiping the surface with one of the RTU cleaning agents mentioned below.
3. Disinfect; liberally wipe the surface again with RTU wipes (avoid pooling of the disinfectant).
4. Wipe dry after recommended dwell time (1-3 minutes – refer to each mfg. instructions).
5. It is best practice no matter what disinfectant chemistry is used, to wipe the surface with a damp cloth and mild soap and finally a damp cloth (with liberal amounts of water).

Recommended RTU products:

Super Sani Cloth Germicidal Disposable Wipe

3M Hard Surface Disinfectant Cleaner Wipes

Wood Wyant ED Disinfectant

Virox 5

Oxivir TB

Virex[®] II 256 One-step Disinfectant Cleaner & Deodorant

Virex II 128 One-step Disinfectant Cleaner

Noco Spray Sanikleen

ODOR CONTROL: The above products are odor counteractants. If you do not use these products you may have to prevent odors on the cover by cleaning daily with neutral soap mixed in luke warm water. The use of a scented cleaner / disinfectant can also be an option.