



Loupes: Disinfecting and Cleaning Instructions

1. Disinfecting

- Purchase standard pure cotton balls
 - Readily available in drug stores, Walmart, Costco, and others
- Use 70% Isopropyl Alcohol only
- Dampen a cotton ball with the Alcohol. Do not pour on or saturate
- Wipe down the loupes

2. Cleaning and polishing

- Disposable lens cloth
- Commercial lens cleaner (made for glasses)
- Dampen the disposable cloth with lens cleaner. Do not saturate
- Polish the lenses of your frame

Warning:

- Do not put your loupes under water
 - Water does not disinfect anything!
- Do not use other disinfecting agents
 - Other disinfecting agents have not been tested on the specialized coatings on your new lenses
 - The lenses have special coatings for
 - Anti-glare
 - Anti-fog
 - Anti-scratch
- Do not use Kleenex, paper towels, or 2X2 gauze/cotton squares
 - These can contain particles that can scratch your optics
- Do not place your loupes in an autoclave or ultrasonic cleaner

Headlight: Disinfecting and Cleaning Instructions

1. Disinfecting

- Purchase standard pure cotton balls
 - Readily available in drug stores, Walmart, Costco, and others
- Use 70% Isopropyl Alcohol only
- Dampen a cotton ball with the Alcohol. Do not pour on or saturate
- Wipe down the surface areas of the headlight
- Wipe down the headlight cable as needed

WARNING:

If you have a non-curing filter, DO NOT clean with any form of disinfectant, including isopropyl alcohol. This will damage the non-curing coatings on the filter.