
Folding Commode

Silla inodoro plegable/
Chaise d'aisance pliante

FOLDING COMMODE DIAGRAM

Silla inodoro plegable/Chaise d'aisance pliante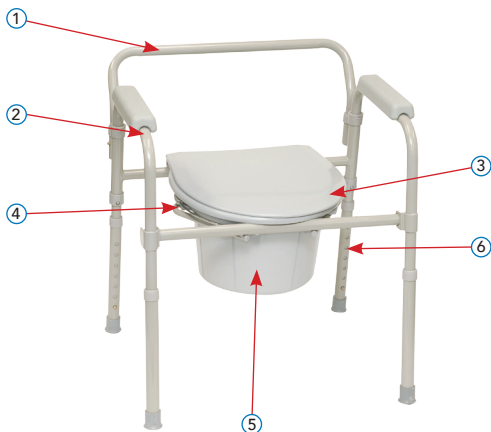

WEIGHT CAPACITY:
350 lb/159 kg (BSFC)

Capacidad de peso / Capacité de poids

	English	Español	Français
1	Backrest support	Soporte de respaldo	Support du dossier
2	Side frame with contoured armrest	Estructuras laterales con apoyabrazos contorneados	Châssis latéral avec accoudoir profilé
3	Toilet seat and lid	Asiento y tapa del inodoro	Siège et couvercle de toilette
4	Bucket support frame	Estructura de soporte del cubo	Cadre du support à seau
5	Bucket with handle, splash guard, and lid	Cubo con asa, protector para salpicaduras y tapa	Seau avec poignée, pare-éclaboussures et couvercle
6	Height adjustable legs	Patas de altura regulable	Pattes à hauteur réglable

Congratulations on purchasing a quality ProBasics® Folding Commode. Our commodes have a versatile design that can be used as a bedside commode, raised toilet seat or toilet safety frame. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the commode's benefits and your safety.

DO NOT ATTEMPT TO ADJUST OR OPERATE THIS COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.



ASSEMBLY INSTRUCTIONS

Remove the folding commode components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the commode. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.


NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS



1. Remove frame, seat, lid, pail, cover and splash shield from carton. Allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled onto commode back frame rail.
2. Hold the folded frame in the upright position.
3. Unfold the two (2) side frame pieces and stand in the upright position (Fig A).
4. Adjust the sides frame pieces so they are slightly wider than perpendicular to the back of the frame (Fig B).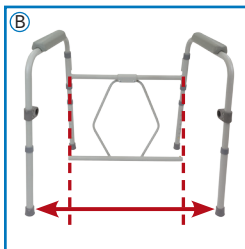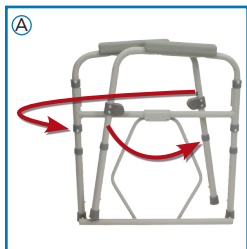

- Swing the bucket support frame up and attach the front of the tube to the 2 side frame pieces. Insert the tube by depressing the brass push pin and pushing until it is securely in place. When secure in place, the brass push pin will extend out of the locking hole. An audible click will be heard. Repeat this step for other side (Fig C).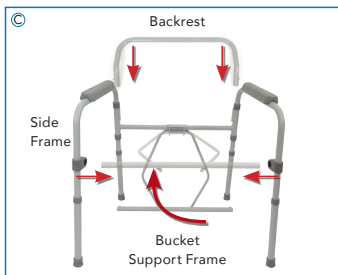

- Attach backrest by depressing brass push pins and sliding either end into the receptacles located on the backside of the frame. Ensure brass push pins fully protrude out of the locking hole. An audible click will be heard once locked.
- Place the commode pail between the pail brackets on the commode frame (Fig D).
- Press the commode seat and lid on to the back tube of the bucket support frame. Commode seat **MUST** be installed before use. Ensure the bucket lies flat on the frame. Remove the bucket lid before use. (Fig E).

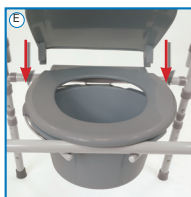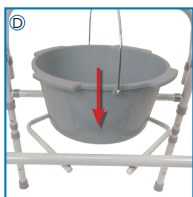ADJUST COMMODE HEIGHT

- Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole. An audible click will be heard.
- Repeat for other three leg attachments.
- To ensure legs are locked into place, pull gently on each leg.

CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

TO USE COMMODE OVER TOILET

(AS A RAISED TOILET SEAT OR TOILET SAFETY FRAME)

1. Set commode to desired height (see "Adjust Commode Height" section).
2. Position splash guard in place onto pail supports.
3. Remove back support.
4. Lift toilet bowl seat and place commode over toilet with splash shield positioned over the center of the toilet bowl.
5. Lower the commode seat onto the frame before use.

NOTE: Clean seat and splash shield with disinfectant after each use.

TO USE AS A BEDSIDE COMMODE

1. Set commode to desired height (see "Adjust Commode Height" section).
2. Push commode pail onto pail supports. Ensure that pail is properly positioned below commode opening. Commode is now ready for use.
3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

CAUTION: It is **NOT** recommended to remove the backrest support when using as a bedside commode. Serious injury may result.

MAINTENANCE

1. Use a mild soap and water solution, rinse thoroughly with clean water, then dry with a soft cloth. Ensure that commode is well rinsed and dried before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of commode.
3. Check fit and tightness of all nuts, bolts and knobs **WEEKLY** to ensure that the commode is stable and safe to use.
4. Check the rubber tips on the height adjustable legs for rips, wear or if they are missing. Immediately discontinue use and replace any or all parts if imperfections exist.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE THE COMMODE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 350 lb/159 kg (BSFC).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- The toilet seat must be in down position before use. Failure to do so may result in injury.
- Ensure the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure the leg extensions are locked in position and an even height is achieved.
- **DO NOT** remove the backrest when using as a bedside commode. Serious injury may result.
- **DO NOT** stand on the commode seat or frame.
- All four leg extensions with rubber tips must touch the floor at all times.
- Users with limited physical strength should be supervised or assisted while using commode.
- Always use caution when floor is wet. Ensure armrest handles are dry before use.
- Use the armrest for assistance only. **DO NOT** use the armrests to support full body weight.
- Weight should be evenly distributed when using the commode. **DO NOT** lean or reach for objects while on the commode.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in commode areas even when using support rails.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

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LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

COMPASSHEALTH

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800) 376-7263

www.compasshealthbrands.com
42-BSFC_FoldingCommode_01 ©2017