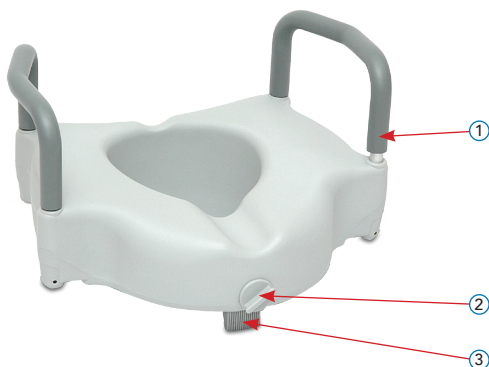
Raised Toilet Seat

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*The arms are included with item #BSRTSLA only.

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Congratulations on purchasing a quality ProBasics® Raised Toilet Seat. Our raised toilet seats offer stability, comfort and easy installation. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the raised toilet seat's benefits and your safety.

DO NOT ATTEMPT TO ADJUST OR OPERATE THIS RAISED TOILET SEAT WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS

Remove the raised toilet seat and any components from all packaging including tape, cardboard and plastic. Check all parts for damage before use. If any parts are missing or damaged, **DO NOT** attempt to use the raised toilet seat. If you do not understand the manual, contact your dealer for a complete explanation of the instructions, cautions and warnings. Otherwise injury may result.

NOTE: DO NOT use excessive force when installing any parts.

SETUP INSTRUCTIONS

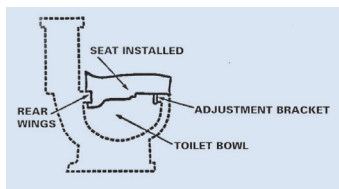
NOTE: For best results, clean and sanitize raised toilet seat as well as the surfaces of the toilet bowl prior to installation.

Model BSRTS - Raised Toilet Seat

1. Lift toilet lid and seat assembly and place rear wings of raised toilet seat around rim of toilet bowl (some toilet bowls may require that seat assembly be removed to allow proper fit).

NOTE: The front adjustment bracket must clear the rim in order for the raised toilet seat to fit securely.

2. With raised toilet seat resting on rim, straighten and center product over toilet bowl as necessary.
3. Check for secure fit routinely before using product. User weight should be evenly distributed over raised toilet seat, not leaning forward, back or to the sides as the seat may tip.
4. If the raised toilet seat does not properly fit your toilet, it is recommended to discontinue use immediately.

Model BSRTSL/BSRTSLA - Raised Toilet Seat with Lock (with Arms)

1. Lift toilet lid and seat assembly and place rear wings of the raised toilet seat in place under the toilet bowl lip at the back of the toilet bowl (some toilet bowls may require that seat assembly be removed to allow proper fit).

NOTE: The front adjustment bracket must clear the rim in order for the raised toilet seat to fit securely.

2. Rotate the raised toilet seat down until the front of the seat rests on the toilet bowl.
3. If necessary, rotate the adjustment knob counterclockwise until the bracket fits inside the toilet bowl.
4. Turn the adjustment knob clockwise to tighten the bracket against the inner surface of the toilet bowl.
DO NOT overtighten.
5. Check the stability of the raised toilet seat. Realign and/or adjust as necessary. When the raised toilet seat is properly installed, the seat will sit flat on top of the bowl and fit securely against the inner rim of the bowl.
6. For model BSRTSLA (with arms), install by depressing both push buttons on the arm and insert into the mounting tubes. An audible click will be heard. Repeat for the other arm.

MAINTENANCE

1. Clean regularly with a toilet sanitizer or household cleaning products.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of raised toilet seat.
3. Check regularly to make sure the raised toilet seat is securely locked onto the toilet.
4. Regular inspection for deformation, corrosion, breakage, wear or compression is recommended. Replace the raised toilet seat if any of these conditions exist.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR USE THE RAISED TOILET SEAT WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Maximum weight capacity: 350 lb/159 kg (BSRTS, BSRTSL, and BSRTSLA).
- **DO NOT** use this device if it is not securely installed.
- This product should not be used without instruction from a healthcare professional.
- Ensure the adjustment knob is tightened and secure at all times.
- Use the arms for assistance only (BSRTSLA model only). **DO NOT** use the arms to support full body weight.
- Ensure **ALL** screws, nuts and/or bolts are tightened and threading is intact on a weekly basis.
- Weight should be evenly distributed when using the raised toilet seat. **DO NOT** lean or reach for objects while seated on the raised toilet seat.
- **NEVER** stand on the raised toilet seat.
- It is recommended that the user ease down onto the seat. **DO NOT** sit on the seat with direct force.
- Always use caution when floor or surfaces are wet.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- Users with limited physical capabilities should be supervised or assisted in shower or tub areas even when using support rails.
- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- Consult a healthcare professional for advice and instructions specific to your needs before use.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.

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LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

COMPASSHEALTH

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800) 376-7263
www.compasshealthbrands.com