
Adjustable Folding Cane

ADJUSTABLE FOLDING CANE DIAGRAM

Bastón plegable y regulable/Canne ajustable pliante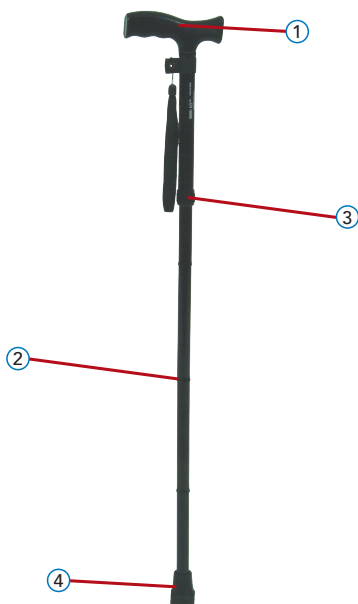

WEIGHT CAPACITY :
300 lb/136 kg (CNFBK and CNFBZ)

	English	Español	Français
1	Hand grip	Empuñadura	Poignée
2	Push button	Botón	Goupille-poussoir
3	Locking ring	Anillo de bloqueo	Anneau de verrouillage
4	Cane tip	Punta del bastón	Embout de la canne

OPERATING INSTRUCTIONS

To ensure the safe operation of ProBasics® Canes, these instructions must be followed.

PROPER CANE HEIGHT

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.

NOTE: For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane.

HEIGHT ADJUSTMENT

1. Loosen the locking ring by twisting clockwise.
2. Adjust the cane height by depressing the brass push button and sliding the lower section upward or downward to desired height.

NOTE: An audible click will be heard when push button securely protrudes through the adjustment hole.

3. Repeat steps 1-2 until cane is at proper height.
4. Tighten locking ring by turning counter-clockwise.

MAINTENANCE

1. Ensure the snap button releases and re-engages properly.
2. Inspect cane tips for wear and damage.
3. Replace any broken, damaged or worn items immediately.

WARNINGS

DO NOT ATTEMPT TO ADJUST OR OPERATE CANE WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- The cane is intended to be used solely as a walking aid and all other use can cause risk of injury.
- This product has a maximum user weight limitation of 300 lb/136 kg. Canes are **NOT** designed to support the total weight of individuals.
- Before use, ensure cane tip is properly seated and straight. Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
- Before use, the height adjustment push button **MUST** be fully engaged and protruding through height adjustment hole on cane to ensure a positive lock. **ALWAYS** test to see that cane handle is properly and securely locked in place or a fall could result, causing bodily injury and/or damage to the cane.
- Make sure locking ring is securely tightened at all times.
- This cane is **NOT** intended for use on stairs.
- Avoid wet floors, ice, soft ground and similar hazards.
- If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts.
- Failure to follow these instructions may result in serious injury.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

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LIMITED LIFETIME WARRANTY

This ProBasics® product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to precise standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to our customers for years of dependable service.

Your ProBasics® brand product carries a limited lifetime warranty on the frame and 90 days on all other non-durable components including but not limited to rubber accessories, and grips, which are subject to normal wear and need periodic replacement. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty does not cover product failures due to normal wear and tear or device misuse. This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

COMPASSHEALTH

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800) 376-7263
www.compasshealthbrands.com
42-CNFBK_CNFBZ_00 ©2017