
DreamEasy 2[™]
Full Face
CPAP Mask

Mascarilla Cara completa CPAP/
Masque Visage complet de CPAP

FULL FACE CPAP MASK DIAGRAM

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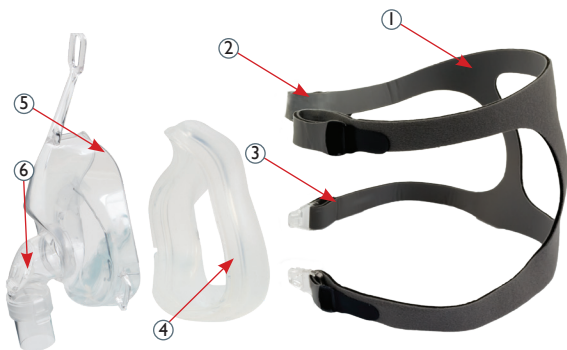* The DreamEasy 2™ Full Face CPAP Mask Starter Kit includes all three (3) sizes of the Full Face Comfort Cushion. One (1) each of size Small (S), Medium (M) and Large (L). The medium (M) cushion is pre-installed on the mask for your convenience.

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OPERATING INSTRUCTIONS

Thank you for choosing the DreamEasy 2™ Full Face CPAP Mask. This user guide provides instructions and guidance on properly fitting and maintaining your new mask.

⚠ CAUTION: Please make sure you read **ALL** instructions and warnings prior to attempting to assemble or operate this CPAP mask. Otherwise, injury or damage may result.

INDICATIONS FOR USE

The DreamEasy 2™ Full Face CPAP Mask is intended to be used by adult patients who have been prescribed CPAP or Bilevel therapy by a physician. It is used in conjunction with positive airway pressure (PAP) by means of a noninvasive device that channels airflow to the patient. The mask is intended for single-patient reuse in the home, hospital or institutional environment.

CAUTION

US Federal Law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

This mask may not be suitable for patients with the following conditions:

- Excessive reflux
- Impaired cough reflex
- Hiatal hernia
- Impaired cardiac sphincter function

DO NOT use if the patient is:

- Unresponsive
- Obtunded
- Unable to remove the mask on their own
- Uncooperative

FITTING YOUR MASK

DREAMEASY 2 FULL FACE MASK — STARTER KIT

NOTE: If you have purchased a size specific mask and **NOT** the starter kit, please skip to the next section.

1. Using the Full Face Mask Fitting Guide provided, align your nose into the small, medium and large cutouts. The correct size for you should align with the top bridge of your nose down to your lower lip, just between the lip and chin. The widest parts of your nose and lips should not touch either side of the cutout.
2. Once the cushion size has been determined, you can proceed to fitting the mask. The medium seal comes pre-attached to the mask. If that's the correct size, you may skip to the next section.
3. To change the cushion to another size, simply hold the mask shell with clean hands and gently pull the Comfort Cushion off and set it aside.
4. Remove the desired size seal from its package and hold it so the size indicator is on the bottom and facing away from the mask shell. With the mask shell in your other hand, align the three (3) notches (near each of the three (3) corners) on the seal to the three ridges on the shell. Gently push the seal onto the mask shell until it is firmly attached.

DREAMEASY 2 FULL FACE MASK — SIZE SPECIFIC

NOTE: If you have purchased the starter kit and **NOT** a size specific mask, please make sure you read the previous section before you proceed.

For your convenience, the headgear and Comfort Cushion have already been attached to the mask.

1. To put the mask on, unhook the headgear clip from the lower left or lower right side of the mask shell.

2. Slip the headgear and mask over your head and place the cushion over your nose in a comfortable position. **NOTE:** The seal should not obstruct your nostrils or breathing in any way. If there is any obstruction, check the cushion size and or mask position.

3. Re-hook the lower headgear clip. The mask may feel loose on your face.

4. To tighten, start with the top two (2) straps. Using both hands, release the hook and loop tabs on each side, of the headgear. Gently pull the tabs back until the top of the mask shell conforms to your head. Reattach the hook and loop tabs, ensuring they are firmly in place. **DO NOT** overtighten*.

5. Repeat with the bottom two (2) straps. The Comfort Cushion should conform to your facial contours. **DO NOT** overtighten*.

6. Lie down and readjust the straps if needed.
7. Connect your PAP device's air tubing to the rotating elbow on the front of the mask. Ensure the other end of the tubing is attached to the device. Turn on your PAP device and use as directed by your physician.

REMOVING YOUR MASK

1. Unhook one or both of the lower headgear clips from the mask shell.
2. Gently pull the mask up and away from your face going over your head.

CLEANING YOUR MASK AND HEADGEAR

Both the mask shell and headgear should be hand washed with a mild soap and warm water. The seals can also be cleaned with an alcohol-free mask wipe to remove excess facial oils. **DO NOT** use abrasive cleaners such as bleach to clean your mask. Ensure that all components are thoroughly rinsed and air dried before using again. **DO NOT** store your mask in direct sunlight.

- It is recommended that the Comfort Cushion be replaced regularly, as often as twice per month.
- It is recommended that the complete mask and headgear are replaced every 3 – 6 months.
- **ONLY** use Roscoe DreamEasy 2™ replacement parts.

* The forehead pad and Comfort Cushion™ **SHOULD NOT** be so tight that the mask frame is pushing into your face or forehead. There should be wiggle room for movement.

WARNINGS

DO NOT ATTEMPT TO SET-UP, ADJUST OR OPERATE THIS CPAP MASK WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- This mask is **NOT** suitable for providing life support ventilation.
- Keep **ALL** vent holes of the exhalation port clear.
- **DO NOT** overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- The mask should **ONLY** be used with PAP devices (CPAP, Auto PAP, Bilevel) as prescribed by a physician or respiratory therapist.
- **DO NOT** use the mask unless the PAP device is turned on and operating properly. Once mask is correctly fitted and the device is turned on, the exhaled air will be flushed out while allowing fresh air in. Using the mask without the device turned on can result in rebreathing exhaled air and in some circumstances, could lead to suffocation.
- Follow **ALL** precautions and physician instructions when using supplemental oxygen in conjunction with PAP therapy.
- Oxygen supports combustion. Oxygen **MUST NOT** be used while smoking or in the presence of an open flame.
- **ONLY** use oxygen in well ventilated rooms/areas.
- If using supplemental oxygen with PAP therapy, **ENSURE** the oxygen flow is turned **OFF** when the PAP device is not in use. Unused oxygen can accumulate within the device or tubing, increasing the risk of fire.
- **DO NOT** smoke or light a flame in the presence of oxygen.
- Discontinue use of mask if any adverse reaction occurs. Consult your physician immediately.
- Contact a health care professional if chest discomfort, shortness of breath, stomach distention, belching or severe headache are experienced.
- **AVOID** eating or drinking three hours before using this mask to minimize the risk of vomiting during sleep.
- **DO NOT** use this mask if you are taking drugs that may cause vomiting.

- Using this mask may cause soreness in the jaw or aggravate an existing dental condition. If symptoms persist, contact a physician or dentist immediately.
- Ensure **ALL** packaging has been removed before using the mask.
- **DO NOT** use bleach, chlorine, alcohol or aromatic-based solutions, these may cause hardening and reduce the life of the product. They may also cause irritation to the skin.
- If **ANY** components (mask, headgear, cushion) appear to be deteriorating at **ALL** (cracking, discoloration, tearing), discard and replace immediately.
- Some rebreathing may occur at low CPAP or Bi-level pressures due to inadequate air flow to clear all exhausted gas from the tubing.
- Refer to your CPAP or bilevel device manual for details on settings and operational information.
- Discontinue use and consult your healthcare provider or physician if the following occurs:
 - Mask discomfort or irritation
 - Allergic reaction
- **⚠ WARNING:** This product can expose you to chemicals including BPA, which is known to the State of California to cause birth defects or other reproductive harm. For more information, visit www.P65Warnings.ca.gov.

STORAGE

For optimal results, make sure to clean and thoroughly dry your mask before storing it for any period of time. Keep out of direct sunlight.

DISPOSAL

This mask may be disposed with normal trash.

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90 Day Limited Warranty

This warranty is solely for the benefit of the original consumer purchaser.

Your Roscoe brand product is warranted to be free of defects in materials and workmanship for 90 days after original purchase date.

This device was built to exacting standards and carefully inspected prior to shipment. This 90 Day Limited Warranty is an expression of our confidence in the materials and workmanship of our products.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. This warranty does not extend to nondurable components, which are subject to normal wear and need periodic replacement.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands dealer.

Record this information to retain for future reference:

Registre esta información para conservarla para referencia futura:
Enregistrer cette information pour conserver pour référence future:

Purchased from/ Comprado de/ Acheté de:

Purchased on/ Comprado en/ Acheté le: ___ / ___ / ___

Lot No./ Numero de lote/ Numéro de lot:

COMPASSHEALTH

Manufactured for

Fabriqu  pour / Fabricado por:

Compass Health Brands Corp.

Middleburg Heights, OH 44130

www.compasshealthbrands.com

800.376.7263 • Made in Taiwan

Fabriqu  en Taiwan / Hecho en Taiwan

41-CPM-DENS_03   2018