DO NOT RETURN THIS PRODUCT TO STORE

We thank you for purchasing this Carex Health Brands Product. At Carex, we strive to bring you superior products that will satisfy the highest of standards for Quality, Durability and Performance. Any questions or concerns about any aspect of this product need to be directed to the toll free number below. **DO NOT RETURN THIS PRODUCT TO STORE.** We are available Monday – Friday, 8AM – 5PM CST, to address your needs. You can reach us Toll-Free at: 1-800-526-8051

Or you can write to us at:
Carex Health Brands
Attn: Customer Service
PO Box 2526
Sioux Falls, SD 57101-2526

You may also visit our website: WWW.CAREX.COM for more information about this or any other Carex Health Brands product.

Carex Health Brands
© 2010
PO Box 2526
Sioux Falls, SD 57101-2526
Phone: 800-328-2935
Fax: 888-616-4297
www.carex.com

B316-00 and B317-00

toilet seat elevator

with removable safety handles

Assembly and Installation Instructions

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- 1**
Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.

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- 2**
To assemble handles:
Note: handles are marked for right and left

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b. Depress the brass buttons on the handle while sliding the handle into position. You will hear “clicks” when the buttons are positioned properly. (You will see the brass buttons pop through the position holes.)

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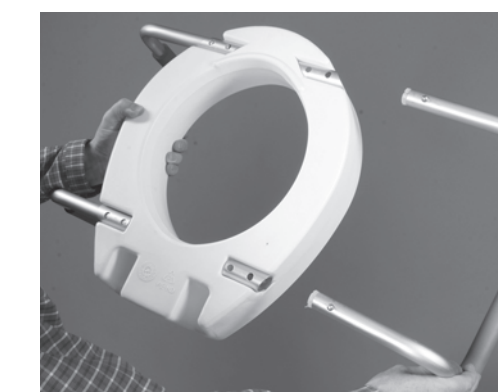
To remove handles:

d. To remove the handles, depress the brass buttons while pulling out on the handles.

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Installation / Instalación

- 3**
Remove existing toilet seat and lid from toilet.

Quite el asiento y la tapa originales del inodoro.
Note: For most toilets you will need to reach behind and underneath toilet to unscrew the nut while holding screw with screwdriver.

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- 4**
Position toilet seat elevator on rim of toilet bowl and center.

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- 5**
Place original toilet seat and lid on top of toilet seat elevator with holes lined up.

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- 6**
Use screws provided to attach original seat, lid and toilet seat elevator to bowl. Secure with nuts provided.

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- 7**
Check for secure fit routinely before using product.

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- 8**
User's weight should be centered over toilet seat.

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