# Reclining Wheelchair

Silla de ruedas reclinable/Fauteuil  
roulant inclinable

# RECLINING WHEELCHAIR DIAGRAM

Silla de ruedas reclinable/Fauteuil roulant inclinable

## WEIGHT CAPACITY:

300 lb/136 kg (WCR1616E, WCR1816E, and WCR2016E)

400 lb/ 181 kg (WCR2218E)

Capacidad De Peso / Capacité Poids

	English	Español	Français
1	Neck Support Cushion	Cojín Para Soporte Del Cuello	Coussin Appui-Tête
2	Back Upholstery	Tapicería De Respaldo	Dossier Rembourré
3	Padded Armrest	Apoyabrazos Acolchado	Accoudoir Rembourré
4	Wheel Locks	Trabas De Las Ruedas	Verrous De Sécurité Des Roues
5	Front Rigging	Aparejo Frontal	Appui-Pieds Fixe
6	Foot Plate	Placa Para Los Pies	Repose-Pieds
7	Headrest Extension	Extensión De Reposacabezas	Extension De L'appui-Tête
8	Hand Grip	Empuñadura	Poignée
9	Seat Rail	Riel De Asiento	Rail Du Siège
10	Hand Rim	Aro De Propulsión	Main Courante
11	8" Caster	Rueda De 8 In	Roulette De 8 Po

# OPERATING INSTRUCTIONS

Congratulations on purchasing a quality ProBasics® Reclining Wheelchair. Our wheelchairs combine style and durability at an affordable price. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the wheelchair's benefits and your safety.

## **DO NOT ATTEMPT TO ADJUST OR OPERATE THIS WHEELCHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

### **TO OPEN CHAIR**

Open wheelchair by tilting to one side and pushing down on seat rail until fully opened.

### **TO CLOSE/FOLD CHAIR**

1. Fold footplates to vertical position.
2. Grab seat with both hands at front and back edge.
3. Tilt chair to one side and lift up on seat rail.

**NOTE:** Keep hands and fingers clear of moving parts to avoid injury.

### **REMOVABLE ARMS**

Remove arms by depressing spring button and lifting arms out of front and rear sockets.

### **HOW TO ATTACH FOOTRESTS/LEGRESTS**

1. Position a footrest so it is pointing 90 degrees away from the reclining chair (footrest platform should be pointing forward when folded down at this point).
2. Align the holes with the footrest pegs on the frame and gently press down until the footrest can swing freely.
3. Swing the footrest until the footrest platform is pointing inward. Continue with this rotation until the footrest locks in place. An audible click will be heard.
4. Repeat steps 1 - 3 with the other footrest.

### **ADJUSTING LENGTH OF FOOTREST/LEGREST**

**NOTE:** When adjusting length of footrests or legrests, user should be sitting in wheelchair with feet on the foot plates.

1. Loosen bolt on the clamp around support assembly.
2. Raise or lower footplate to desired position that will provide maximum support of thighs.

**NOTE:** Front rigging should not be so low that it causes uncomfortable pressure to thighs at front of seat upholstery.

**NOTE:** Footrest should not be lower than 2½" from the ground to ensure proper clearance.

## ADJUSTING ELEVATING LEGREST

1. Lift footplate on legrest assembly to desired position.
2. Release legrest assembly and it will automatically lock into place.
3. To lower, push out on legrest lever and lower footplate to desired position.

## REMOVAL OF ELEVATING LEGRESTS

1. Release lock by pushing on the front rigging lever.
2. Swing-away footrest/legrest out and sideways, and lift to remove.

## ADJUSTING SEAT HEIGHT

**DO NOT ATTEMPT!** Any adjustment to your product seat can affect the stability of your product. We highly recommend that you have a dealer or licensed technician adjust your seat height to ensure adjustments are safe.

## WHEEL LOCKS

**NOTE:** Engage both wheel locks before getting in or out of chair, leaning forward in chair, or while in an elevator or wheelchair lift. Locks are engaged by pushing the handles completely forward.

**NOTE:** Proper adjustment of wheel lock is at least 1/8" when locked.

1. Adjustments to locks are made by loosening or tightening the nut or the carriage bolt.
2. Embed lock shoe at least 1/8" by sliding clamp toward rear wheel while handle is engaged in locked position.
3. Tighten nut and bolt to secure in position.
4. Test for correct locking action before actual use.

## ANTI-TIPPER INSTALLATION

1. Press the release buttons and remove the anti-tippers from the wheelchair frame.
2. Reinsert the anti-tippers with the wheels curved down towards the ground/floor until they lock back into place.
3. Press the release buttons on the lower portion of the anti-tipper and slide it up or down to achieve a 1 1/2" to 2" ground clearance.
4. Make sure both anti-tippers are adjusted to the same setting and release buttons are fully engaged before use.

# MAINTENANCE

## GENERAL CARE

- Wipe chrome parts with a clean, soft cloth at least once a week.
- All metal parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately.
- Do not use an abrasive cleanser that will scratch the finish.

## WHEELS

- Large rear wheels should be inspected at least once a month to ensure proper tightness.
- If either wheel wobbles or is loose, release lock nut and tighten axle bolt slightly.
- Test for proper tightness and alignment by leaning chair to one side and spin adjusted wheel.

## TIRES

- Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth.
- Replace if tires become severely worn or cracking appears.

## FRONT CASTERS

- Casters should be checked periodically and lubricated occasionally.
- Lubricate by removing fork and greasing caster stem bearings.
- Lubricate wheel bearings by removing wheel from fork.
- Reassemble fork and wheel by tightening nuts until no play remains, but wheels roll freely.
- If nut is too loose, caster will shimmy.
- If it is too tight, chair will be difficult to steer.

## SEAT UPHOLSTERY

- Wash the upholstery at least once a month with soapy water
- Confirm that the upholstery is dry before using.

## HAND GRIPS

- Use only a clean towel dampened with cool water when cleaning hand grips.
- Confirm that the grips are dry before using.
- **NEVER** use ammonia-based cleaning solutions or soap on hand grips. This may cause the hand grips to loosen or slide off the frame.

## WARNINGS

### **DO NOT ATTEMPT TO ADJUST OR OPERATE WHEELCHAIR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- Maximum Weight Capacity:  
300 lb/136 kg for WCR1616E, WCR1816E, and WCR2016E;  
400 lb/181 kg for WCR2218E
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind. Always transfer wheelchair user to a vehicle manufacturer's approved seat and use safety restraints available.
- **DO NOT** operate this wheelchair on roadways, streets, or

any surfaces with vehicular traffic or on uneven surfaces such as sand, gravel, hills, ice or dirt roads.

- **DO NOT** use the wheelchair on stairs or escalators.
- **DO NOT** attempt to operate on inclines 10° or greater without proper assistance.
- **DO NOT** attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily injury or damage to the wheelchair.
- Be sure to completely engage wheel locks when getting in and out of wheelchair or when in elevator or wheelchair lift.
- Wheelchair should be on a smooth, stable and level area with wheel locks engaged before transferring user in or out of wheelchair.
- **DO NOT** adjust footrests any lower than 2 ½" from floor. When getting in or out of wheelchair, do not stand on or apply weight to footrests. Make sure the footplates are in the up position when getting in or out of the chair.
- Before reaching or leaning forward, rotate casters to front of the wheelchair and engage wheel locks to help prevent wheelchair from tipping over.
- **DO NOT** lean over or apply weight on the top of wheelchair back. This can cause wheelchair to tip over. When trying to extend arms over back, reach only as far as your arm can extend without having to alter your seating position.
- **DO NOT** stand on wheelchair.
- **DO NOT** use the wheelchair without the anti-tipping supports installed. Anti-tipping supports should always be 1 ½" - 2" off the floor.
- **DO NOT** use wheel locks to slow your descent down an incline. Abruptly changing directions while going down an incline could cause instability or possible tipping over.
- **ALWAYS** use the handrims – **DO NOT** grip the rear wheels themselves.
- **DO NOT** attempt to lift the wheelchair by any detachable components.
- **DO NOT** tip the wheelchair without assistance.

**USE CAUTION. FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION.**

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# LIMITED LIFETIME WARRANTY

This warranty is solely for the benefit of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and an 18-month warranty from date of purchase on non-durable components, which are subject to normal wear and need periodic replacement. Compass Health Brands warrants the side frames and crossbraces for the lifetime of the original consumer purchaser. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to nondurable components, including but not limited to rubber accessories, casters, grips and upholstery, which are subject to normal wear and need periodic replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, or any circumstance beyond Compass Health Brands' control, such as improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

**COMPASSHEALTH**

6753 Engle Road  
Middleburg Heights, Ohio 44130  
Tel.: (800) 376-7263

[www.compasshealthbrands.com](http://www.compasshealthbrands.com)

42-WCR\_RecliningWheelchairs\_00 ©2017