Two Button Walker With 5" Wheels and Seat

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Walker Parts Included:

2 Leg Extenders
+ 2 Leg Extenders
w/Wheels

WKA AW2BST

WEIGHT CAPACITY: 300 lb (136 kg)

Capacidad de peso / Capacité de poids

	English	Español	Français
1	Contoured hand grips	Empuñaduras contorneadas	Poignées profilées
2	Push button	Botón	Goupille-poussoir
3	Seat storage hooks	Almacenaje del asiento Ganchos	Crochets de stockage de siège
4	Foldable seat	Asiento plegable	Siège pliable
5	Seat hooks	Los ganchos de asiento	Crochets de sécurité
6	Height adjustable legs	Patas de altura regulable	Pattes à hauteur ajustable
7	Glide leg tips	Puntas de la pata	Embouts
8	Height adjustable front wheels	Ruedas delanteras de altura regulable	Roues avant à hauteur ajustable

Congratulations on purchasing a quality ProBasics® Two Button Walker with Seat. Our two button walkers are lightweight and convenient in providing stability and balance in your daily activities. Please be sure to read these instructions and warnings thoroughly and follow the instructions of your healthcare provider to maximize the walker's benefits and your safety.



DO NOT ATTEMPT TO ADJUST OR OPERATE THIS WALKER WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

ASSEMBLY INSTRUCTIONS


Remove all packaging including tape, cardboard and plastic before use. Lay out all the pieces and make sure to inspect them prior to assembly. If any parts are missing or damaged, **DO NOT** attempt to assemble the walker. Contact your distributor immediately.

NOTE: DO NOT use excessive force when installing any parts.


1. Hold the folded frame in the upright position (Fig. A).
2. Unfold the two (2) side frame pieces until you hear an audible click from each of the push buttons (Fig. B). Stand the walker in the upright position. Gently push the side frame pieces towards the center to ensure the push buttons have securely locked into place.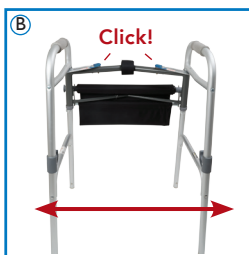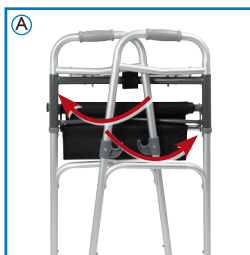

ASSEMBLY INSTRUCTIONS (CON'T)

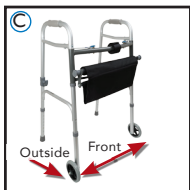3. When installing the wheel legs on your two button walker, you must make sure they are installed on the front two legs and the wheels are facing the outside of the walker (Fig. C).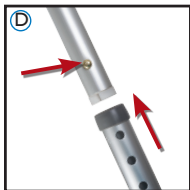

4. Insert one (1) leg tip into the end of one of the back legs of the side frame (Fig. D). Insert the leg tip by depressing the brass push pin and sliding it until it is securely in place. When the leg tip is securely in place, the brass push pin will click into place and extend out of the locking hole. Repeat this step for the other leg tip.

5. Insert one (1) wheel leg into the end of one of the front legs of the side frame (Fig. D). Insert the wheel leg by depressing the brass push pin and sliding it until it is securely in place. When the wheel leg is securely in place, the brass push pin will click into place and extend out of the locking hole. Repeat this step for the other wheel leg.
6. Make sure the two (2) front wheels are facing to the outside of the walker (Fig. C).


USING THE WALKER SEAT

1. On a flat surface, remove the seat from the front seat storage hooks (Fig. F).
2. Pull the seat edge back and place into the front seat hooks (Fig. G).
3. Ensure the bar is securely locked into place before use.
4. When the seat is not in use or during transport, it should be placed on the storage hooks (Fig. F).

ADJUSTING THE HEIGHT

Before this two button walker can be used, the handle height will need to be adjusted to the user's specific needs. Once the walker has been opened, legs are installed and the sides have been confirmed as locked into place, stand with the middle of your feet even with the walker's back legs.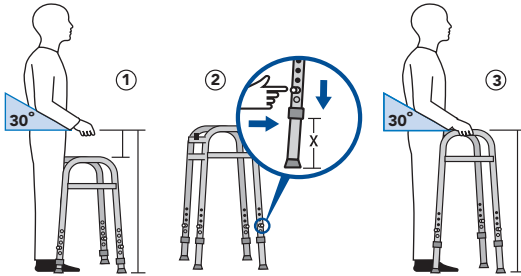

1. With arms relaxed at your side, the handle of the walker should be at wrist height. Or with your hands on the hand-grips, your arms should be flexed 20° – 30°.
2. Depress the brass push pins and slide each leg until the desired height is reached. Once the desired height has been reached, lock the leg securely in place by sliding the leg until the brass push pin clicks into place and extend out of the locking hole. Repeat this step for the other three legs. The lock holes have been numbered to ensure all four legs match the same height.
3. To be sure the legs are securely locked into place, gently push down on the hand grips or pull on each of the legs.

CAUTION: Ensure that all four walker legs are adjusted to the same height and that **ALL** brass push pins are extended through the locking holes before use.



WALKING WITH THE WALKER

1. Make sure the seat is in the storage position.
2. Start with the middle of your feet even with the walker's back legs.
3. Holding the hand grips, pick the walker up and place it in front of you at an arms length that feels comfortable. Be sure all four legs return to the floor.
4. Carefully step forward with your weaker leg into the middle of your walker followed by your stronger leg. This will help keep weight off your weaker leg by using your arms for added support.

FOLDING THE WALKER

This two button walker is designed to fold quickly and easily for transportation and storage.

1. Make sure the seat is in the storage position.
2. Standing behind the walker, press down and hold the blue button on the right side with your left hand (Fig. G).
3. With your right hand, grab the right side rail and swing it all the way towards the center of the walker (Fig. H).
4. Repeat steps 2 and 3 for left side of walker, until completely folded (Fig. I).

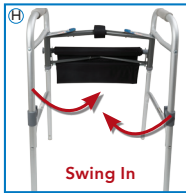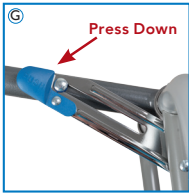MAINTENANCE

- To clean, use a mild soap and water solution, rinse with clean water on a damp cloth, then dry with a soft cloth.
- Ensure that the walker has been well dried before using.
- **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of the walker.
- Check all the parts and movable connections before each use to ensure that the walker is safe to use. If deemed unsafe, immediately discontinue use and replace any or all parts in question.
- Check the rubber tips on the height adjustable legs for rips, wear or if they are missing. Immediately discontinue use and replace any or all parts if any of these imperfections exist.
- To ensure the best service from your product, always use ProBasics® replacement parts. Contact your authorized dealer for more information.

WARNINGS

- **DO NOT ATTEMPT TO ADJUST OR OPERATE THIS WALKER WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

WARNINGS (CONT'D)

- Maximum weight capacity: 300 lb (136 kg).
- Make sure your walker is adjusted to your correct height.
- **DO NOT** stand on walker or seat.
- Inspect all parts for shipping damage before assembly. If there is shipping damage — **DO NOT USE**. Contact dealer/carrier for further instructions.
- This product should not be used without instruction from a healthcare professional.
- Do not use this walker in pools, showers or other water environments, as this could result in corrosion of parts that result in damage to the walker and possible injury to the user.
- Always avoid unsafe walker conditions, such as soft, slippery or moving surfaces, uneven paths and obstacles.
- Do not exceed the maximum weight capacity intended for this product.
- If you are transferring to or from a wheelchair to use this product, be sure the wheelchair wheels are securely locked.
- This walker is not intended for use on stairs.
- Do not push one side of the walker ahead and then other side without lifting the walker first.
- This walker is **NOT** equipped with rear wheel brakes, **DO NOT** install the 5" wheels on the back legs.
- 5" wheels **MUST** be installed on the front legs only with the wheels facing the outside of the walker.
- Make sure that the brass push buttons on all the parts are fully protruding through the locking holes. This will ensure that all connections are locked in position.
- Check the rubber tips on the height adjustable legs for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- Accessories designed by other manufacturers have not been tested by Compass Health Brands and are not recommended for use with this product.
- **DO NOT** use the hand grips to fully support the user's body weight.
- Users with limited physical capabilities should be supervised or assisted when using the walker.
- Compass Health Brands assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**FAILURE TO FOLLOW THE
ABOVE INSTRUCTIONS MAY RESULT
IN SERIOUS INJURIES.**

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LIMITED LIFETIME WARRANTY

This device was built to precise standards and was carefully inspected prior to shipment. Your ProBasics® brand product is warranted to be free of defects in materials and workmanship, has a Limited Lifetime Warranty on the frame and a one year warranty from date of purchase on all other parts, which are subject to normal wear and need periodic replacement. If within such warranty period, the product proves defective, the product will be repaired or replaced, at the option of Compass Health Brands. This warranty does not cover product failures due to normal wear and tear or device misuse.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty provides only for replacement of defective parts and does not cover shipping or labor charges. Compass Health Brands' sole obligation and your exclusive remedy under this warranty is limited to such repair and/or replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond Compass Health Brands' control, or improper operation, maintenance or storage.

If you have a question about your Compass Health Brands product or this warranty, please contact an authorized Compass Health Brands Dealer.

6753 Engle Road
Middleburg Heights, Ohio 44130
Tel.: (800) 376-7263

www.compasshealthbrands.com

42_WKAAWBST_TwoButtonWalkerwWheels_00 ©2017