BIO-C[®] TEMP

ENGLISH

DEFINITION

Ready-to-use bioceramic paste for intracanal dressing.

INDICATIONS

1. Intracanal dressing for endodontic treatment in teeth with pulp necrosis and retreatments;
2. Intracanal dressing in teeth with perforations, external and internal resorptions, prior to the use of MTA ANGELUS or MTA REPAIR HP;
3. For the treatment of incomplete root formation (rhizogenesis).

COMPOSITION

Calcium Silicates, Calcium Aluminate, Calcium Oxide, Calcium Tungstate and Titanium Oxide.

TECHNICAL INFORMATION

- Biocompatible product ready for use;
- High alkalinity - pH: 12±1;
- High radiopacity: ≥ 9 mm in the aluminum step-wedge, according to the ISO 6876 standard.

USE TECHNIQUE

A. USE OF APPLICATOR TIPS

1. Sterilize the applicators tips before use;
2. Place the applicator tip at the syringe;
3. We recommend using a disposable plastic cover on the syringe to avoid cross-contamination;
4. Confirm the output of the material from the syringe before the direct application to the canal;
5. Position the tip and syringe assembly properly into the canal and press the plunger with little force to prevent the excessive output of the product;
6. Before closing the syringe, retract the plunger to stop the outflow;
7. Clean any excess material in the tip of the syringe and close with appropriate pressure after use to prevent moisture input, which causes the product to dry;
8. Clean and disinfect the syringe in subsequent uses to avoid cross-contamination.

B. INTRACANAL APPLICATION

1. Anesthetize, install rubber dam isolation and then perform the biomechanical preparation of the canal and, in case of internal resorption, use specific instruments to remove the inflammatory tissue;
 2. Dry the canal with absorbent paper points;
 3. Discard the material of the beginning of the syringe, as it may be a little hard.
 4. Connect the applicator tip to the syringe;
 5. Insert the tip of the syringe near the dental apex;
 6. Apply BIO-C[®] TEMP through gradual retraction of the syringe to obtain a complete filling of the canals;
 7. Remove any excess paste from the pulp chamber;
- ① *Note: Make the changes if necessary, until signs and symptoms normalize.*
8. The final removal of Bio-c Temp before sealing should be performed using Sodium Hypochlorite associated with the use of a K Type File making grinding movements against the root canal walls;
 9. Then, irrigate with 17% EDTA solution and if possible activate EDTA with an Ultrasonic Tip in 3 cycles of 10 seconds duration;
 10. Seal the canal with a sealer such as BIO-C[®] SEALER or MTA-FILLAPEX and gutta-percha cones;
 11. Seal the coronary opening.

WARNINGS

- The product is sensitive to moisture, so properly close the packaging with adequate pressure to prevent dryness;
- Do not store it in the refrigerator;
- Do not reuse applicator tips;
- Clean and sterilize the syringe in subsequent uses to avoid cross-contamination.

PRECAUTIONS

- Wear safety goggles, face shield and gloves when handling the product. In case of contact with eyes or skin, wash with water.
- Do not use the product after its expiration date.
- Do not use the product if its packaging is damaged.

- Do not use it in patients that report sensitivity to the components of the formula.
- Product not tested in pregnant and lactating women.
- Keep the product away from children.

STORAGE

- Keep the product in a dry and ventilated place between 15 and 30°C and with relative humidity below 60%.
 - Do not store it in the refrigerator.
- * *Product to be used exclusively by the dental surgeon.*

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