

Composi-Tight® 3D Fusion™

Sectional Matrix System

from **Garrison**
Dental Solutions



When design and function come together...you get FUSION!

Composi-Tight® Sectional Matrix System

- ✓ Operator-friendly retaining system
- ✓ Naturally contoured bands
- ✓ Anatomically correct contacts
- ✓ Contacts at the height of contour
- ✓ Rings produce optimal tooth separation for excellent, tight contacts

An all-new system designed to handle even the toughest cases

More than 20 years of sectional matrix leadership and an incredible amount of feedback from users all over the globe were poured into the design. All this hard work has resulted in separator rings with significantly improved performance and ease of use.

Hugs the curves Soft-Face™ silicone tips with enhanced marginal ridge anatomy hug the matrix band to the tooth for flash free restorations

It just keeps going...


Enhanced durability for extended ring life and greater tooth separation

Get a grip!

All Composi-Tight® 3D Fusion™ separator rings incorporate Ultra-Grip™ retention extensions that anchor them securely to virtually any tooth. These rings stay where you place them!


Stack it up - Tru-Stack™

technology allows placement of the orange ring (tall) over the blue ring (short) for easier MOD and multiple tooth restorations

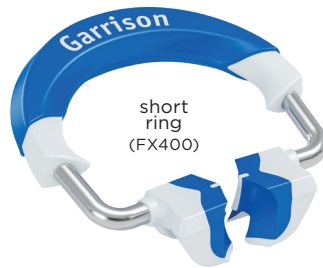


Matrix band perfection

Composi-Tight® 3D Fusion™ Full Curve Non-Stick Matrices are three dimensionally contoured, ultra-thin and dead-soft to help you recreate ideal interproximal anatomy.

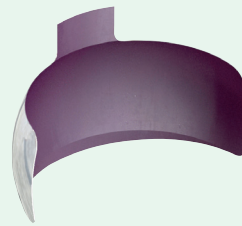

Easy in - easy out! A Grab-Tab™ centered on the occlusal edge of the band simplifies placement while Slick Bands™ non-stick coating reduces composite adhesion by an impressive 92%!

Go deep or wide - Improved subgingival extensions and added wraparound length make for the perfect matrix band to use with the green Wide Preparation ring. Now you're ready to tackle even the widest preps!

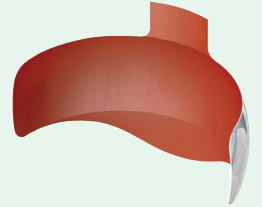

Dentistry's first Wide Preparation separator ring (green) makes Composi-Tight® 3D Fusion™ the most versatile and user-friendly sectional matrix system available. The Wide Prep ring greatly simplifies what until now has been one of the most challenging posterior composite restorations. Now you can restore these big preps with confidence knowing that you will achieve excellent contact and contour without having to take extra steps.

Ultra Adaptive Wedges

Combining Soft-Face™ adaptive materials, a firm inner core and advanced mechanical features to produce a wedge that truly works. Every time.

Fins with serious function

Soft retentive fins smoothly fold down during wedge insertion and then spring back when clear of the interproximal space. Wedge back-out is a thing of the past.



A real softie Composi-Tight® 3D Fusion's Soft-Face™ overmold allows the wedge to do what no other wedge can truly do - actually adapt to interproximal irregularities.

Composi-Tight® 3D Fusion™

Sectional Matrix System

from **Garrison**
Dental Solutions

RESTORATIVE

MATRIX SYSTEMS



Only Composi-Tight® 3D Fusion™ can make the technique this intuitive and frustration free!

Tight contacts - guaranteed!

Superior tooth separation force through advanced engineering and material selection guarantees perfect contacts every time.

Ideal anatomy

Soft-Face™ silicone ring tips with enhanced marginal ridge anatomy hug the pre-contoured dead-soft matrices to the tooth for flash free, perfectly contoured restorations.

Simplified technique

Composi-Tight 3D Fusion is packed with features like easy ring stacking, over the wedge placement and Ultra Grip anti-ring slip extensions that make system use intuitive and frustration free.

Place the matrix band:




Dead-soft three dimensionally contoured matrices have placement tabs for easier occlusal placement and Garrison's revolutionary non-stick coating for hassle free removal.

Wedge firmly:



Composi-Tight® 3D Fusion™ Ultra-Adaptive wedges glide in but won't back out. Their soft, flexible exterior readily adapts to interproximal irregularities for a superior gingival seal.

Place the ring:

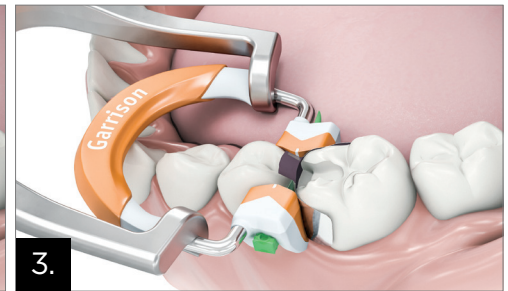

All three of Composit-Tight® 3D Fusion's separator rings will fit right over the wedge for easier placement. With Ultra-Grip™ retention extensions, they will stay firmly in place from the distal of the canine to the widest of molar preparations.

Full Curve Matrix Bands - Refills

Composi-Tight 3D Fusion				
FX100 - 4.0mm w/ext 4.4 mm 50 or 100/pk	FX150 - 4.1mm w/ext 6.0 mm 30 or 60/pk	FX175 - 5.0mm w/ext 5.6 mm 50 or 100/pk	FX200 - 5.4mm w/ext 6.6 mm 50 or 100/pk	FX300 - 6.0mm w/ext 8.7 mm 30 or 60/pk

Ultra Adaptive Wedges - Refills

Composi-Tight 3D Fusion			
FXYL 100/pk	FXBL 100/pk	FXOR 100/pk	FXGR 100/pk

Full Curve bands wrap further around the tooth, out of the way, making ring placement easier. Dead-soft and a skinny 0.0016" thin, they're the perfect choice.

Combining Soft-Face adaptive materials, a firm inner core and advanced mechanical features to produce a wedge that truly works. Every time.

3D FUSION MATRIX KIT FX-KFF-10	203101
1 x Short Ring (blue) 1 x Tall Ring (orange) 1 x Wide Prep Ring (green) 70 x Assorted Matrix Bands 80 x Assorted Ultra-Adaptive Wedges 12 x Assorted FenderWedges 1 x Ring Placement Forceps	
3D FUSION MATRIX SYSTEM FXR-KFF-10	203100
1 x Short Ring (blue) 1 x Tall Ring (orange) 1 x Wide Prep Ring (green) 70 x Assorted Matrix Bands 80 x Assorted Ultra-Adaptive Wedges 4 x Assorted FenderWedges 1 x Ring Placement Forceps 12 x Rally Polishers	
3D FUSION WEDGE KIT 200PK	202905
3D FUSION SHORT RINGS BLUE 2PK	203103
3D FUSION TALL RINGS ORANGE 2PK	203104
3D FUSION WIDE PREP RINGS GREEN 2PK	203105
3D FUSION RINGS KIT (ONE OF EACH) 3 PK	203107



TEACHING BETTER DENTISTRY

1-800-265-3444
clinicalresearchdental.com