

# HYGIENIST PINK STRIP





from **CONTACTEZ**

HYGIENE

The ContactEZ Hygienist Strip is an advanced, high-precision dental strip designed to remove extrinsic discoloration and plaque from interproximal surfaces without causing damage to tooth enamel. The non-abrasive, flexible, stainless steel strips clean proximal contact areas of tough coffee, tea, tobacco, and red wine stains quickly, safely, and easily to ensure patient satisfaction and save hygienist chair time.

- Removes stains and plaque from proximal surfaces
- Non-abrasive perforated strip protects enamel
- Access areas that are difficult to reach with other tools
- Leaves a smooth proximal surface for flossing
- Flexible, highest-quality, stainless steel strip
- Thinner than dental floss
- Autoclavable, steam heat only, up to 270°F

## The Stain Remover

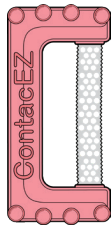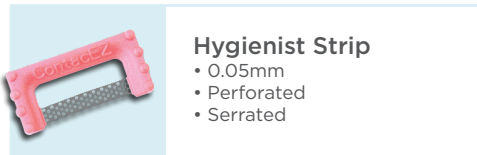

Actual Size

## Objective

Removing dark stains from interproximal surfaces can be an exceptionally difficult task. Stubborn stains, plaque, and calculus can set in tooth surfaces that are problematic to access, and abrasive tools and prophylactic pastes can damage the enamel from a patient's teeth.

ContactEZ Hygienist Strips are a new solution to these problems. Thinner than dental floss, these single-handed, ergonomic strips are devised to fully and efficiently clean proximal contact surfaces while leaving the enamel unharmed.



## Method

The final aesthetic appearance of a patient's smile depends on a complete and thorough cleaning and polishing of all tooth surfaces.

Insert the Hygienist Strip into the interproximal space using two fingers. Carefully pass the strip buccolingually, back and forth, with gentle force, flexing the strip mesially and distally as necessary until the targeted stain is removed.

- Removing all stains and plaque from the interproximal surface ensures a completely clean and appealing look for the patient.
- Proper polishing helps reduce pitting and defects on proximal surfaces which can cause accumulation of staining agents.
- The ContactEZ Hygienist Strip provides optimal results, saves chair time, and ensures patient satisfaction.


**CONTACTEZ**  
Hygienist Pink Strip  
8-pack

208279


# HYGIENIST PINK STRIP

from **CONTACEZ**<sup>®</sup>


CLINICAL TECHNIQUE

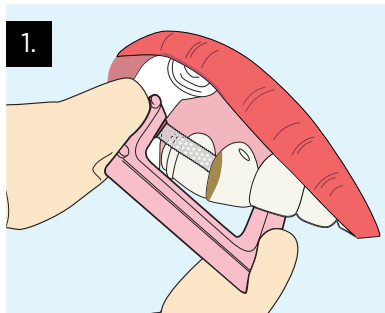

1. Place a cotton roll between the lip and gum. Insert the strip into the interproximal space by passing back and forth buccolingually with minimal vertical pressure.

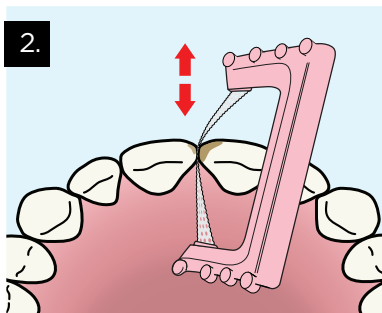

2. Use gentle force to curve the flexible strip along the stained interproximal surface. Pass the strip back and forth buccolingually, maintaining the pressure on the stained surface until the discolouration has been removed.

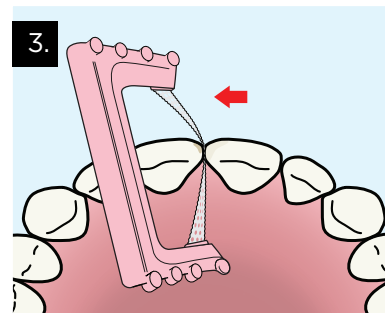

3. Repeat as necessary on the opposite tooth. The strip will curve and clean along the natural curvature of the tooth with its built-in flexibility. The non-abrasive perforated strip will not damage enamel, just remove the stain.



Protect the lip by using either a cotton roll or fingers to move it out of the way. Insert the strip with minimal vertical pressure.



Passing the thin, perforated strip back and forth buccolingually will even remove stains between tight contacts.