

IPR STRIP SYSTEM

from
CONTACTEZ



The ContacEZ IPR Strip System is a safe and innovative interproximal reduction system that makes interproximal enamel stripping efficient and accurate without creating sharp corners or subgingival ledges. Designed for use with Invisalign®, ClearCorrect™, eClinger, Six Month Smiles®, and other orthodontic treatments, these flexible diamond strips curve and conform along the natural contours of the teeth to maximize patient comfort and safety.

- Ergonomic, single-handed design
- Provides optimal tactile control
- Will not remove excessive enamel
- Will not cut lips or gums
- Prevents sharp corners or subgingival ledges
- Effective for anterior and posterior areas
- Central opening for maximum visual perception
- Saves chair-time

ContacEZ IPR Strip System




Yellow IPR Starter • 0.07 mm

- Single-Sided
- Extra-Fine Diamond
- Serrated

Start to open the contact safely & easily with a serrated edge.

(208303)

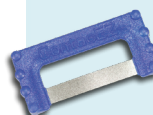

Dark Blue IPR Widener • 0.15 mm

- Double-Sided
- Coarse Diamond

Widen interproximal spaces and reduce enamel efficiently.

(208305)



Red IPR Opener • 0.12 mm

- Double-Sided
- Medium Diamond

Open the contact and reduce enamel gently on two teeth.

(208304)



Green IPR Extra-Widener • 0.20 mm

- Double-Sided
- Coarse Diamond

Further widen interproximal spaces safely for ideal alignment.

(208306)

ContacEZ IPR Plus Strips



Clear IPR Single-Sided Opener • 0.10 mm

- Single-Sided
- Medium-Fine Diamond
- Serrated

Reduce enamel on the proximal surface of one tooth.

(208275)



Purple IPR Super-Widener • 0.25 mm

- Double-Sided
- Coarse Diamond

Further widen the interproximal space between two teeth.

(208307)



Cyan IPR Single-Sided Widener

- 0.15 mm
- Single-Sided
- Medium-Coarse Diamond

Further reduce enamel on the proximal surface of one tooth.

(208276)



Brown IPR Mega-Widener • 0.30 mm

- Double-Sided
- Coarse Diamond

Create an extra-wide interproximal space between two teeth.

(208308)

IPR STRIP SYSTEM

from
CONTACTEZ



RESTORATIVE

Usage Guidelines & Technique

This technique is especially effective for anterior teeth and for case refinement. ContacEZ IPR Strips are autoclavable (**steam heat only, up to 270°F/132°C for 10 minutes at 30 psi max**) and reusable, and should be sterilized prior to the first use. **The manufacturer recommends to use each strip 3-5 times for maximum efficiency and functionality.**



- 1 Hold the strip between your thumb and forefinger, insert the strip into the interproximal space, and gently pass the strip back and forth buccolingually until the strip is passive. Do not use vertical force. Start with the thinnest diamond strip, and repeat with thicker strips until you have reduced (stripped) the correct amount of enamel according to your treatment case prescription.
- 2 To ensure accurate measurements, polish the proximal surface using the ContacEZ Gray Final Polishing Strip.
- 3 Verify the width of the gap created using the ContacEZ Incremental Thickness Gauge. The location and amount of any interproximal reduction performed should be recorded in the patient's chart.
- 4 **Pediatric dentists** can use ContacEZ IPR Strips for the serial proximal reduction of primary molars.

ASSORTED REFILL 8-PACK 208268

2x Yellow IPR Starter, 2x Red IPR Opener, 2x Dark Blue Widener, 2x Green IPR Extra-Widener

COMBINED 40-PACK 208265

8x Yellow IPR Starter, 8x Red IPR Opener, 8x Dark Blue IPR Widener, 8x Green IPR Extra-Widener, 2x Clear IPR Single-Sided Opener, 2x Cyan IPR Single-Sided Widener, 2x Purple IPR Super-Widener, 2x Brown IPR Mega-Widener

ASSORTED 32-PACK 208266

8x Yellow IPR Starter, 8x Red IPR Opener, 8x Dark Blue Widener, 8x Green IPR Extra-Widener

COMBINED INTRODUCTORY 8-PACK 208267

1x Yellow IPR Starter, 1x Red IPR Opener, 1x Dark Blue IPR Widener, 1x Green IPR Extra-Widener, 1x Clear IPR Single-Sided Opener, 1x Cyan IPR Single-Sided Widener, 1x Purple IPR Super-Widener, 1x Brown IPR Mega-Widener

YELLOW REFILL 8-PACK 208269

RED REFILL 8-PACK 208270

DARK BLUE REFILL 8-PACK 208271

GREEN REFILL 8-PACK 208272

IPR PLUS STRIP ASSORTED 32-PACK 208281

8x Clear IPR Single-Sided Opener, 8x Cyan IPR Single Sided Widener, 8x Purple IPR Super-Widener, 8x Brown IPR Mega-Widener

IPR PLUS STRIP ASSORTED 8-PACK 208280

2x Clear IPR Single-Sided Opener, 2x Cyan IPR Single Sided Widener, 2x Purple IPR Super-Widener, 2x Brown IPR Mega-Widener

IPR PLUS STRIP CLEAR REFILL 8-PACK 208275

IPR PLUS STRIP CYAN REFILL 8-PACK 208276

IPR PLUS STRIP PURPLE REFILL 8-PACK

SINGLE-SIDED 208277
DOUBLE-SIDED 208307

IPR PLUS STRIP BROWN REFILL 8-PACK

SINGLE-SIDED 208278
DOUBLE-SIDED 208308