

WHITENING

Opalescence®
● ● ● go



Powerful,
Professional
Whitening.
TO GO!

No custom tray, no problem!

Opalescence Go® is the perfect solution for a large demographic of patients, especially those who don't wish to have a custom tray fabricated and want to whiten in just 15 minutes to an hour a day. It's also a great follow-up treatment for those patients who've had in-office whitening with Opalescence Boost, and wish to touch up their smiles - or even as an in-between solution for patients who are waiting for their custom trays to be fabricated.

Patient with braces?

That's no problem either. Opalescence Go's unique Ultra-Fit tray fits directly over orthodontics, too.



Before



After



As a prescription-strength alternative to OTC strips and paint-on formulations, Opalescence Go is formulated to deliver dramatic whitening results, but with patient comfort at the forefront. The Ultra-Fit tray moulds with the heat of the mouth to provide a comfortable, custom-like fit, and the whitening gel can easily be brushed away after whitening.

Available in 10% hydrogen peroxide (with wear times from 30-60 minutes) and 15% hydrogen peroxide (with wear times from 15-20 minutes). Mint flavour.



INSTRUCTIONS FOR USE



1. Remove from package



2. Centre tray on arch



3. Gently suck down on the tray



4. Remove outer tray



5. Suck down or swallow again



6. Wear 10% for 30-60 minutes per day or 15% for 15-20 minutes per day

Opalescence® Go

ACTIVE INGREDIENT:
10% and 15% hydrogen peroxide

CONTAINS:
Potassium nitrate and fluoride

WEAR TIME:
10% 30-60 min/day
15% 15-20 min/day

INDICATIONS:
Professional whitening to go or as an alternative to over-the-counter products

FLAVOUR:
Mint

Whitens more teeth at once!

Unlike strips, Opalescence Go's unique disposable tray provides molar-to-molar coverage, and the optimal gel quantity in the pre-loaded tray ensures the gel comes in contact with more posterior teeth.

Potassium Nitrate + **Fluoride** + **20% Water**

Reduces Sensitivity + For Tooth Health + Keeps Teeth Hydrated

OPALESCENCE GO SINGLE PATIENT KIT	
10 upper and 10 lower trays	
Mint 10% HP	704630
Mint 15% HP	704633
OPALESCENCE GO 6-PACK	
6 patient kits	
Mint 10% HP	704635
Mint 15% HP	704638
SAMPLE TOWER	
20 ready-to-go 10% sample trays	
	704901