

WHITENING

Opalescence[®] PF
tooth whitening systems



Share a
Bright,
Beautiful
Smile

The #1 Doctor Recommended Professional Tooth Whitening System*

While PF could quite easily stand for “Patient Friendly”, it actually means “Potassium Nitrate with Fluoride” – and it’s the reason why this patented formulation is clinically proven to lower sensitivity associated with tooth whitening, increase the mineral content of the enamel for greater strength, and reduce susceptibility to caries ^{1,2,3,4,5}.



Before



After

Custom
Take-Home



1-800-265-3444 | CLINICALRESEARCHDENTAL.COM

CRD Clinical Research Dental

Exclusive Distributor






Opalescence has a sticky gel consistency so it stays in contact with the tooth for enhanced whitening.

Teeth treated with fluoride-based whitening gels experienced significant reduction in caries susceptibility when compared to their untreated controls. This is a significant advantage over other whitening systems, including those containing amorphous calcium phosphate or just potassium nitrate alone.

No Shade Rebound

Opalescence PF is specially formulated with 20% water to prevent dehydration of the tooth and subsequent shade relapse – a common problem with many other whitening systems.

Opalescence PF is highly effective in treating internal tooth discolourations due to congenital, systemic, pharmacologic, and traumatic factors, and aging. You'll also see great success with fluorosis and tetracycline staining, although the results will take longer.

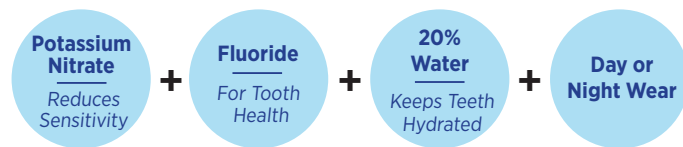

Gel stays where it should - for maximum effectiveness!

Opalescence PF's sticky, viscous formula means it won't leach or seep from the tray like other bleaching gels. That means there's less risk of it being ingested, or of it leaching and causing possible irritation to, and sensitivity of, the soft tissue. The sticky formulation also ensures Opalescence PF stays in contact with the teeth to deliver maximum bleaching effectiveness.

Opalescence PF is available in a mild mint flavour, as well as regular (no flavour). Other whitening products contain poly glycol, which because of its bitter taste, must be masked with strong flavours like cherry or peppermint. This strong flavouring causes the mouth to create excess saliva, and is uncomfortable for overnight use. This excess saliva will also dissolve or wash the gel out of the tray – leaving less than ideal results!



* Based on global sales data.

RESEARCH:

- 1 Al-Qunaiyan TA, The Effect of Whitening Agents on Caries Susceptibility of Human Enamel, Operative Dentistry, 2005.
- 2 Rodrigues JA, Basting RT, Serra MC, Rodrigues AL, Effects of 10% carbamide peroxide bleaching materials on enamel microhardness, American Journal of Dentistry, Vol. 14, No. 2, April 2002.
- 3 ref. Mokhikis GR, Matis BA, Cochran MA, Eckert GJ. "A clinical evaluation of carbamide and hydrogen peroxide whitening agents during day time use." JADA 2000 Sept; 131 (9): 1269-1277).
- 4 L.M. Clark, N. Barghi, J.B. Summitt, B.T. Amaechi, Enamel Fluoride Uptake from Fluoridated Carbamide Peroxide Bleaching Gel, AADR Abstract #0498, 2006
- 5 W.D. Browning, Report on Low Sensitivity Whiteners, AADR Abstract #1650, 2006

Opalescence® PF 10%

ACTIVE INGREDIENT:
10% carbamide peroxide

CONTAINS:
Potassium nitrate and fluoride

WEAR TIME:
8-10 hours/day

INDICATIONS:
For patients with existing sensitivity; can be worn day or night

FLAVOUR:
Mint and Regular

Opalescence® PF 15% and 20%

ACTIVE INGREDIENT:
15% & 20% carbamide peroxide

CONTAINS:
Potassium nitrate and fluoride

WEAR TIME:
15% 4-6 hours/day
20% 2-4 hours/day

INDICATIONS:
For faster whitening; can be worn during the day

FLAVOUR:
Mint and Regular

Opalescence® PF 35%

ACTIVE INGREDIENT:
35% carbamide peroxide

CONTAINS:
Potassium nitrate and fluoride

WEAR TIME:
30 minutes a day

INDICATIONS:
For shorter wear time and touch-ups

FLAVOUR:
Mint

OPALESCE PF DOCTOR KITS

8 x 1.2ml Opalescence syringes, 1x 1.2ml Ultradent LC Block-Out Resin syringe, 1x Black Mini tip, 2 x Sof-Tray 0.035" 5x5" sheets, 1 x 1 oz Opalescence Whitening Toothpaste, 1 x Tray case, 1 x Shade guide

10% PF Mint	706510
10% PF Regular	706509
15% PF Mint	706513
15% PF Regular	706512
20% PF Mint	706516
20% PF Regular	706515
35% PF Mint	706204

OPALESCE PF REFILLS

40 x 1.2 ml Opalescence syringes

10% PF Mint	701305
10% PF Regular	701306
15% PF Mint	701070
15% PF Regular	701068
20% PF Mint	701071
20% PF Regular	701069
35% PF Mint	706205

OPALESCE PF 4-PACK REFILLS

4 x 1.2 ml Opalescence syringes

10% PF Mint	706540
10% PF Regular	706539
15% PF Mint	706541
15% PF Regular	706542
20% PF Mint	706544
20% PF Regular	706543
35% PF Mint	706545