

QuickSplint®




MAKING A QUICKSPLINT

- 1 Fill QuickSplint uniformly with Affinity™ Quick Bite Registration Material. Also, turn the QuickSplint over and fill the midline with Affinity™ Quick Bite Registration Material on the underside.



- 2 Align QuickSplint on the upper or lower teeth, supporting while the patient bites down completely to seat the QuickSplint.

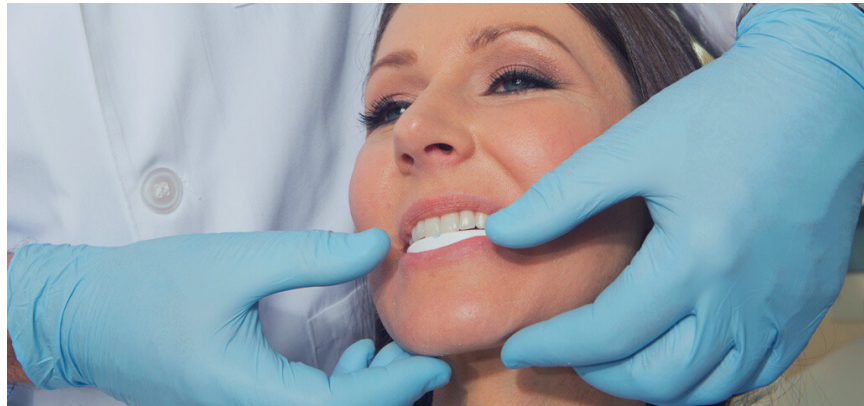

- 3 For upper arch, instruct patient to gently push excess Affinity™ Quick Bite Registration Material with tongue up on the roof of their mouth.

- 4 After Affinity™ Quick Bite Registration Material sets, gently remove the QuickSplint and trim any excess material with a sharp blade. QuickSplint is now ready for use!



CLINICAL TIPS

- TIP 1:** Adhesives are not required.
- TIP 2:** Overfill the tray! Underfilling the tray is the biggest mistake.
- TIP 3:** Make sure the patient bites down to level the tray on the opposing arch.
- TIP 4:** You can reline the QuickSplint during treatment phases.

