

A.S.A.P.[®] Dailies

All Surface Access Diamond Polishers



Polish for a Day and Throw Away

Take esthetics to a new level of efficiency and convenience with A.S.A.P.[®] Dailies, the next generation of daily-use polishing from Clinician's Choice[®].

Designed for multi-use in a single day, A.S.A.P. Dailies flexible spiral shape provides all surface access polishing and a fast, high gloss luster on all direct and indirect restorations and all restorative materials including: composite, ceramic, zirconia, lithium disilicate and provisionals. Delivering consistent results after multiple sterilization cycles,* diamond impregnated A.S.A.P. Dailies reduce your cost-per-use without sacrificing performance – simply polish for a day and throw away!

With no messy polishing paste required, the first step in the A.S.A.P. Dailies polishing sequence is a 15-20 second polish with the purple, 45 micron diamond impregnated Pre-polisher, followed by a light wiping motion with the peach Final High Shine Polisher (3-6 micron diamond particles) for a brilliant shine in as little as 30 seconds.

*Minimum 3 sterilization cycles.

CLINICAL TECHNIQUE

Dentistry and photography courtesy of Dr. Bob Margeas



1. Upper right central composite veneer after pre-contouring with Contours™ Finishing and Polishing Discs.



2. Using a light wiping motion, the 45-micron diamond impregnated Pre-polisher is applied to the restorative surface.



3. The Final High Shine Polisher (peach) is applied for 15-20 seconds. A high luster shine is the result.



4. Final restoration utilizing Evanesce™ Universal shade B2 and A.S.A.P. Dailies.

A.S.A.P. DAILIES ALL SURFACE ACCESS
DIAMOND POLISHERS 50-PACK 133160
Contains: 50 Polishers (25 x Pre-Polishers, 25 x
Final High Shine Polishers)