



## Beautiful Access, Practical Isolation, in a Comfortable Way!

The new Umbrella™ soft tissue retractor was developed to make patient comfort its key component. See the benefits that go beyond a simple retraction of the soft tissues:

- Naturally and gently keeps the mouth propped open, which means no pulling or stretching of the lips
- Provides relief to gaggers, as it doesn't initiate the gag reflex for most patients
- Can be kept in place when checking the bite, or useful in conjunction with a bite block, while the tongue comfortably rests behind the tongue guard
- Adapted for a variety of procedures like class II restorations, sealants, orthodontic bonding, whitening, etc.
- Opens the space between the lips and cheeks in a way that encourages saliva to accumulate away from dentition—thus improving access for HVE
- Easily placed by you or the patient

### Step 1.



Simply press the tabs on the Umbrella™ soft tissue retractor together to prepare it for insertion.

### Step 2.



Ask the patient to place the tip of their tongue on the roof of their mouth.

### Step 3.



Choose one side of the mouth in which to start, and then comfortably insert the other side of the retractor into the cheek.

### Step 4.



Use the tabs to center the retractor with the patient's mouth.

### Step 5.



Check that the patient's tongue is resting comfortably behind the guard, ensuring easy access.

UMBRELLA RETRACTOR MEDIUM 5-PACK	794870
UMBRELLA RETRACTOR MEDIUM 20-PACK	794871
UMBRELLA RETRACTOR MEDIUM 40-PACK	795162