

DualForce™

Sectional Matrix System




Tighter Contacts. Better Seal.

DualForce™ is a complete matrix system designed for use with most Class II composite restorations.


DualForce Rings

Featuring unique dual nickel titanium wires and independent interproximal separators, DualForce Rings provide significant and balanced separation force and an excellent matrix/tooth seal.

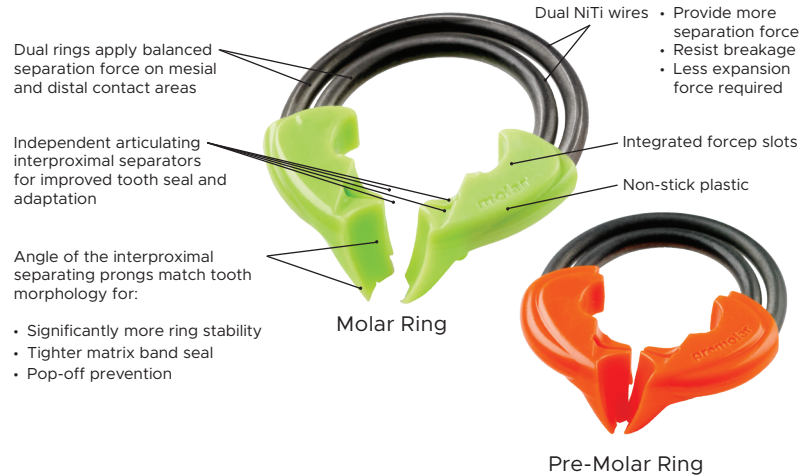


DualForce Active-Wedges™

Designed to enhance separation force, DualForce Active-Wedges™ unique collapsing tip re-opens after placement, locking the wedge in place, and combined with a natural contour ensures a complete marginal seal to prevent overhangs.

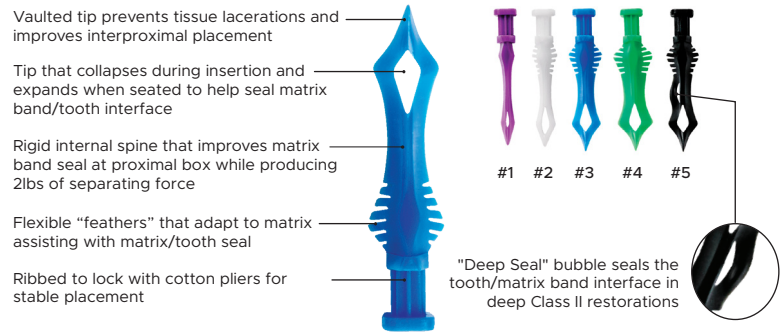

DualForce Ultra-Wrap™ Matrix Bands

DualForce Ultra-Wrap™ Matrix Bands are designed to wrap the tooth snugly to reduce flash and finishing time, while a proximal contour and curved marginal ridge ensures a natural contact position and shape is created every time.

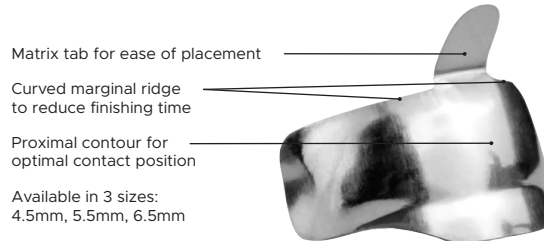

“ The DualForce Sectional Matrix System provides tight contacts and anatomical contours. ”

- Nate Lawson, DMD, PhD

Dentistry and photography courtesy of Parag R. Kachalia, DDS



1 With rubber dam in place, pre-op view of wide Class II on first molar requiring replacement.



2 Ultra-Wrap™ Matrix Band in place extending well beyond the proximal line angles. Ultra-Wrap is made from .0015" semi-hard stainless steel making it more resistant to crimping during placement.



3 The Active-Wedge™ tip collapses during insertion for easy placement, then re-expands when fully seated, regaining full contour for an improved matrix band seal.



4 Once seated, the DualForce™ Sectional Matrix System provides a complete gingival seal with a tight vertical seal of the matrix band along with a strong and stable separation force, resulting in less chance of composite flash.



5 First increment of Evanesce™ restorative material adapted to include Ultra-Wrap's pre-contoured occlusal lip, establishing an ideal rolling marginal ridge.



6 Evanesce composite adapts to the internal surfaces of the preparation with excellent packability.



7 Evanesce is easily sculpted with no slumping, efficiently and effectively establishing the ideal occlusal anatomy before the final curing.



8 Diamond-impregnated A.S.A.P.® Pre-polisher initiates the 2-step polishing sequence in just 10-15 seconds.



9 The A.S.A.P. Pre-polisher gently diminishes small surface defects without affecting anatomy.



10 After just 10-15 seconds of polishing with the A.S.A.P. Final High Shine Polisher, the final esthetic restoration with an optimal proximal contact.

POWERED BY **Hands-OnLine** LIVE

Instant Replay / In a rush? Watch this quick technique tip!

Scan to watch



DualForce™ Matrix Band and Active-Wedge Placement
with Marc Geissberger, DDS, MA





DualForce™ Ring Placement
with Marc Geissberger, DDS, MA



VER. 1.2022 | © Clinician's Choice Dental Products, Inc. 2022. All rights reserved.



STARTER KIT WITH ULTRA-WRAP BANDS 209021

- Contains: 1 x Pre-Molar Ring (orange), 1 x Molar Ring (green), 25 x 4.5mm Ultra-Wrap Matrix Bands, 25 x 5.5mm Ultra-Wrap Matrix Bands, 25 x 6.5mm Ultra-Wrap Matrix Bands, 20 x Active-Wedge #1 (Purple), 20 x Active-Wedge #2 (White), 20 x Active-Wedge #3 (Blue), 20 x Active-Wedge #4 (Green), 20 x Active-Wedge #5 (Black).

COMPLETE KIT WITH ULTRA-WRAP BANDS 209022

- Contains: 1 x Starter Kit (see above for full list of contents), 1 x Punch Pliers, 1 x Ring Forceps.

- RING FORCEPS 209009
- PUNCH PLIERS 209013
- MATRIX FORCEPS 209024

ACTIVE-WEDGE INTRO KIT 209000

- Contains: 40 x each size wedge (#1, #2, #3, #4, #5)

ACTIVE-WEDGE REFILL 100-PACKS

- #1 209001
- #2 209002
- #3 209003
- #4 209004
- #5 209005

- RING 2-PACK REFILLS**
- Pre-molar 209007
 - Molar 209008
 - Assorted 209006

- ULTRA-WRAP MATRIX BAND 100-PACK REFILLS**
- 4.5mm 209018
 - 5.5mm 209019
 - 6.5mm 209020

- ULTRA-WRAP MATRIX BAND TRIAL KIT 209023**