

RESTORATIVE

TrollFoil 4.5

The Gold Standard for Every Dentist!

Smart delivery system allows for dispensing one TrollFoil at a time. Easy to pick up without getting ink on your fingers.

Individual pieces stacked nicely into a little container that fit nicely in any cabinet, drawer or counter top.

- 4.5 Microns thin
- Individual pieces
- Inked on both sides
- Premounted on a frame
- No forceps needed
- Marks on wet & dry surfaces
- Marks on gold & porcelain
- Timesaving in chair
- Extra sensitive to pressure
- Gives a high degree of accuracy at all times



Thinnest articulating product on the market.

TrollFoil 4.5 makes getting those hard to dial in bites easy. No need for patient to return because of the high degree of accuracy all the time.



Distinctive markings on all surfaces

Precise diagnosis, even on wet surfaces, polished surfaces, porcelain and gold. A light tap from the patient and you get the perfect mark.


TrollFoil 4.5 microns is the thinnest articulating foil ever made. Research shows that the thinner the articulating foil, the better. Thick articulating paper generates a less uniform and larger occlusal contact area than the occlusal area marked with thin articulating foil. TrollFoil 4.5 is double-sided blue color mounted on its own frame eliminating the need for forceps. Marks on wet, dry, and highly polished surfaces, at 4.5 microns thick this super-thin foil gives you very precise markings.



1 Comes pre-mounted on its own gray frame no need for forceps. Easily remove 1 TrollFoil from box at a time as needed.



2 Hold TrollFoil with both hands. Your right thumb on the blue sticker and left thumb on the back gray frame near the widest area of the frame.

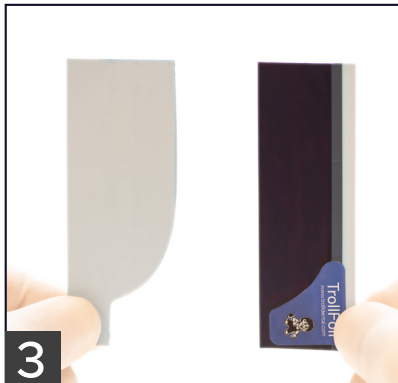

3 Tear away the back grey perforated frame and discard it.



4 Position TrollFoil in patients mouth just before upper and lower arches touch. Instruct patient to gently bite down with only tapping the Foil 2-3 times. TrollFoil is sensitive to pressure since it is a Foil and not a paper.



5 Instruct patient to open mouth and remove TrollFoil. Clearly see distinct markings on upper and lower arches. Marks on both wet and dry surfaces, gold & porcelain.

| | |
|-----------------------------------|--------|
| TROLLFOIL 8 MICRON BLUE 100-PACK | 208318 |
| TROLLFOIL 4.5 MICRON BLUE 80-PACK | 208319 |
| TROLLFOIL 8 MICRON BLUE 500-PACK | 208320 |
| TROLLFOIL 8 MICRON RED 100-PACK | 208323 |