

NEW!

glisten™

PROVISIONAL RESIN GLAZE



Modern provisional composites are very effective at providing ideal form, function and tissue support; however, attempting to polish such materials is very time-consuming and often results in a less than ideal shine. At CLINICIAN'S CHOICE® we believe in making provisional restorations strong, esthetic, functional, and equally as important, efficient to fabricate.

GLISTEN™ provisional polish minimizes your polishing time while providing a beautiful glistening shine on your provisional veneers, crowns, bridges, inlays and onlays. What makes GLISTEN unique is that it can be applied and cured either extra-orally or intraorally due to its resin based formula. GLISTEN can be applied in a very thin layer and once cured makes the provisional "glisten", just like a natural tooth. Compared to other temporary glazing materials, GLISTEN is thinner, can be cured intraorally and with any curing light, and it provides a superior shine.

GLISTEN's shelf life is 2 years.

CLINICAL TECHNIQUE



1. Anterior four unit temporary bridge after trimming with burs and discs.



2. A fine, thin coat of Glisten is applied eliminating the polishing step.



3. When fully cured the temporary is polished, shows less wear, is smoother to the patient's tongue and easier to clean.

Dentistry and photography courtesy of Dr. Robert Margeas.

CLINICIAN'S CHOICE®
DENTAL PRODUCTS INC.
designed for better dentistry

#151207 Glisten 5ml Bottle

1-800-265-3444

www.clinicianschoice.com

**BACKED BY OUR
TRY-IT
GUARANTEE**

We guarantee you'll love GLISTEN, or return it within 30 days for a full refund!