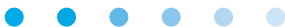
# Opalescence™

tooth whitening systems

FOR **ALIGNERS**

## Instructions

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FR

1.

1. Express one continuous line of gel along the lower half of the cheek side of each aligner tray. Use ¼ to ½ of syringe per arch.

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2.

2. Lightly tap tray to adapt sides to teeth. Ensure that the trays are fully seated and completely engaged onto the aligner buttons. Wear Opalescence 10% gel for 8-10 hours over night.

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3.

3. Remove excess gel with soft toothbrush. Do not swallow rinsed gel.

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4.

4. Clean tray with soft brush and water.

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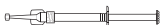
IMPORTED BY/IMPORTÉ PAR:  
 CLINICAL RESEARCH DENTAL, LONDON, ON, N6A 1M6, CANADA

PREVENTION OF DENTAL CARIES/PRÉVENTION DES CARIES DENTAIRES

1.2 ml Opalescence Tooth Whitening/ Blanchiment Dentaire

NPN: 80009556

For an SDS of this product, please see [www.ultradent.com](http://www.ultradent.com)  
 © 2020 Ultradent Products, Inc. All rights reserved.

Manufactured by Ultradent Products, Inc. 505 West Ultradent Drive,  
 South Jordan, Utah 84095 [www.opalescence.com](http://www.opalescence.com)  
 Made in the USA from U.S. and globally sourced materials.

1008789AR02 092920

## Precautions

1. Due to discomfort or sensitivity that can occur as a result of your orthodontic treatment, avoid using this product for the first two weeks after receiving your orthodontic aligners and for the first two days after each new aligner set, as whitening can increase tooth sensitivity.
  2. Limit contact with soft tissue.
  3. Some amalgam fillings may leave a dark purple color in your whitening tray; this is normal.
  4. If you have any questions regarding this product, please consult your dental professional.
  5. Some patients may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional.
  6. Do not eat, drink, or use tobacco products while whitening trays are in place.
  7. Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
  8. For patients or users with allergy concerns, refer to product allergen document available at [www.ultradent.com](http://www.ultradent.com). If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.
  9. Restorative materials will not whiten.
- 
10. Inform your dentist that you have been whitening prior to any dental procedure.
  11. Patients with serious health concerns should consult their primary care provider prior to treatment.
  12. Pregnant or nursing women should not use teeth whitening products.
  13. White spots on the enamel may become more apparent during the whitening process and often blend with continued whitening.
  14. Teeth can re-stain over time, do touch up teeth whitening treatments as necessary.
  15. Avoid contact with eyes.
  16. To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
  17. Monitor progress to prevent whitening beyond the desired shade or translucency.

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