



How Dry is Your Working Environment?

OraSeal® CAULKING & PUTTY

FROM **ULTRADENT** PRODUCTS, INC.




REALITY
Four Star Award



OraSeal caulking and sealing material effectively adheres to a wet rubber dam. Even under water or saliva, OraSeal excels at preventing leakage during treatment. In conjunction with the powder-free, tear-resistant DermaDam, OraSeal makes it easy to achieve a dry working environment.

Achieve a Dry Working Environment

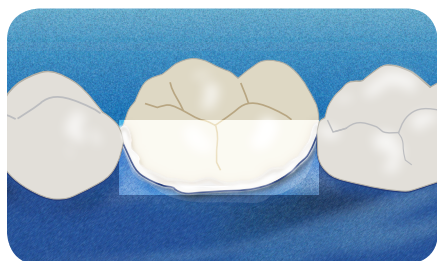

Apply OraSeal Caulking with the Black Mini or White Mac delivery tips to prevent leakage of rubber dam during endodontic treatment*. Shape with wet gloved finger, wet cotton swab or instrument. Root canal is performed in a clean, dry field.



Use OraSeal putty to block out large interproximal spaces, etc., for easy and distortion-free removal of impression.



Use to assure rubber dam seal when using strong peroxide for vital bleaching, or porcelain etching with hydrofluoric acid. Express OraSeal Caulking into embrasures and onto the gingiva. Stretch and position rubber dam onto frame.

*Cohen, S. Burns. Pathways of the Pulp Seventeenth edition, Mosby, 1998 pp 123-124

Ordering Codes:

#715352 - OraSeal Kit

2 x 1.2ml OraSeal Caulking syringes
2 x 1.2ml OraSeal Putty syringes
4 x Black Mini Tips
20 x White Mac tips

#715351 - OraSeal Caulking Refill

#715353 - OraSeal Putty Refill
4 x 1.2ml syringes

#715354 - OraSeal Caulking Econo Refill

#715355 - OraSeal Putty Econo Refill
20 x 1.2ml syringes



1-800-265-3444

www.clinicalresearchdental.com