PermaFlo® Purple
Anatomical Indicating Composite

Description: PermaFlo Purple is a 68% filled, light-cured, radiopaque, thixotropic, methacrylate-based flowable composite with 1µ average particle size. Indications for Use

For immediate endodontic post-op coronal seal and identification of coronal floor and/or canal orifices.

Directions for Use:

- Complete endodontic obturation. Clean pulpal floor with cotton and isopropyl alcohol and dry.
- Apply dentin bonding agent to entire preparation according to manufacturer's instructions.
- Remove Luer Lock cap and attach syringe tip (Black Mini or Micro 20 gauge)
- Verify flow prior to applying intraorally.
- Apply 1-2mm layer of PermaFlo Purple to pulpal floor or over each canal orifice.
- Light cure with VALO® LED Curing Light on Standard Power for 10 seconds. (For lights with output <600mW/cm2 cure 20 seconds or for lights with output >600mw/cm2 cure for 10 seconds).
- Place temporary or permanent restoration.

Precautions and Warnings:

- WARNING:** Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock caps between each use. Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check.
- Carefully read and understand all instructions before using.
- Use only recommended tips.
- Refrigerate PermaFlo Purple when not in use.
- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
- Dispose of tips and empty syringes properly.
- Re-cap syringes with the Luer Lock Cap.
- Disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover to avoid cross contamination.
- Tips are disposable; do not re-use to avoid the risk of cross-contamination.
- Keep out of reach of children.
- Do not use after expiration date noted on syringe.
- For professional use only.

NOTE: For SDS and additional information about using PermaFlo Purple and other related products, please go to www.ultradent.com

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