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 Clinician's Choice®

Quad-Tray® Ultimate™ Posterior

 Manufactured for: Clinician's Choice Dental Products, Inc
167 Central Ave., London, ON Canada N6A 1M6
U.S. Distribution: Brookfield, CT, USA 06804
1.800.265.3444 • 519.641.3066 clinicianschoice.com

Instructions for Use

INTENDED USE: Dental Impression Trays and Accessories intended to be used as a medical device or an accessory to a medical device for the purpose of performing consistent, accurate dental impressions.

Directions For Use

Try-in the Quad-Tray® Ultimate™ Posterior to ensure a proper fit. Adjust lingual wall if necessary. Prepare tooth and proceed with tissue management using a hemostatic agent (Tissue Goo™ Hemostatic Gel) and retraction cord (Re-Cord™ Retraction Cord).

Optionally, apply an EDTA cleanser (Detail™ Pre-Impression Cleanser) to clean and decontaminate the preparation following the manufacturer's instructions for use.

Syringe a light body impression material (Affinity™ Light Body) around the preparation twice in a 360-degree motion, to totally cover tooth preparation, keeping tip moving forward and submerged to avoid air entrapment.

Simultaneously, load Quad-Tray Ultimate Posterior with an appropriate tray material (Affinity™ InFlex Maximum Support Tray Material). Use enough material to encompass the tray walls on the buccal and lingual surfaces. Add additional tray material to the preparation areas on the Quad-Tray Ultimate Posterior.

Seat the tray and instruct the patient to fully, but passively, close. A passive closed position is held for 3 minutes from tray insertion, or until material is fully set.

Remove opposing arch first by holding the tray against the prep while the patient slowly opens. Then use a gentle rocking motion to remove the prep side.

INSPECT THE IMPRESSION:

Prep is completely covered in impression material.

A perfect blend of the tray and wash material is achieved.

The tray is fully encompassed in tray material, with no sign of burn through or distortion.

Recyclability

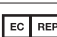
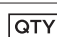
The Quad-Tray Ultimate Posterior is fabricated using aluminum and is 100% recyclable.

Steps to Recycle

Separate the impression material/mesh from the tray and discard.

Recycle the Quad-Tray Ultimate Posterior aluminum frame in the appropriate recycling container, in accordance with your local recycling program.

Symbols Used

	Medical device	 only	Prescription only
	Consult instructions for use		For single use
	CE marking		Catalogue number
	EC authorized representative		Quantity

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