

D•FINE
Shape & ShineTM
 SINGLE STEP DIAMOND POLISHERS

NEW!

**Shape & Contour
 PLUS, Create an
 Unrivalled Polish,
 IN A SINGLE STEP!**




A perennial **REALITY 5-STAR** favorite since its introduction, the CLINICIAN'S CHOICE® D•FINE line of diamond polishers has been expanded to include the new D•FINE **SHAPE & SHINE**TM Single Step Diamond Polishers. These time saving diamond impregnated polishers allow the clinician to effortlessly shape, contour, and polish cured composite material, all in a single step!

D•FINE **SHAPE & SHINE** polishers feature 45 micron size diamond particles that can conservatively contour composite when additional pressure is applied. During polishing, these diamond particles continuously breakdown, getting smaller in just seconds. The breakdown of microns creates submicrons which result in a high luster, unrivalled finish on all hybrid and nano-filled composite materials – and in just seconds! With **SHAPE & SHINE**, slight contouring and polishing adjustments can now be made easily!

D•FINE **SHAPE & SHINE** Single Step Diamond Polishers are durably designed, providing between 6 to 10 uses per polisher, and can be autoclaved at 270°F to 275°F.

Available in 6 shapes to facilitate all restorative requirements.
See technique on reverse.

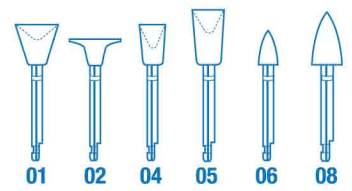

# 415906	Trial Pack 12 polishers, 2 of each shape
# 4155 + 2 digit code of desired shape	Refill Pack 10 polishers, 1 shape

**CLINICIAN'S
 CHOICE**[®]
 DENTAL PRODUCTS INC.

designed for better dentistry

1-800-265-3444

www.clinicianschoice.com

BACKED BY OUR TRY-IT GUARANTEE We guarantee you'll love SHAPE & SHINE, or return it within 30 days for a full refund!

**Shaping, Contouring, and Polishing Composite Restorations
 Using SHAPE & SHINE Single Step Diamond Polishers**

CLINICAL TECHNIQUE



1. Upper right central incisor labial veneer preparation



2. Renamel Nano Light Cured Nanofill Composite veneer adapted and light cured



3. Initial labial lobe form shaping to contour with pointed #08 Shape and Shine



4. To mimic the adjacent central, the Shape & Shine #02 wheel is used to create a flat surface topography



5. Final high luster composite surface created using the Shape & Shine polishers

Dentistry and photography courtesy of Dr. Robert Margas

**CLINICIAN'S
 CHOICE**[®]
 DENTAL PRODUCTS INC.

designed for better dentistry

1-800-265-3444

www.clinicianschoice.com