

1

STEP BY STEP INSTRUCTIONS

Once you open the box your whizzer will look like this. Everything you need is inside. Peel the lid off, no screwing needed.

Attachments (inc bag, strap) sold separately



2

The lid keeps the Whizzer airtight- To extend or compress the whizzer the lid must be **off** and the white stability ring **in** position. Push or pull it out to place from the top. It's durable so can take the force and will loosen after use.

Remove the white stability ring with the attached leakproof funnel to find the unisex spout.



Ensure the highest part is at the front as a privacy shield.

The raised bumps on the spout and funnel fit into the holes on the stability ring. Ensure the parts are secure before use.

(It gets quicker with practice)



Unisex funnel scoops down to fit beneath the body.

Leakproof spout can be removed when in use if fast flow fills it too quickly but replace for carrying when full.

4



Either

Once used, remove the spout, stability ring and funnel all in one to empty immediately. Revert steps back to state 1.

Or

Pinch the spout off and replace with the lid and put both parts in a bag or use the strap to attach it to a bag then empty/ clean later.

Tip:
Keep a few
tissues/wipes
in the bag

We use high-grade food quality silicone so with the right care the Whizzer should last for years of use!

**Clean with hot soapy water after use. *Nothing abrasive. *Dishwasher safe.*

1

STEP BY STEP INSTRUCTIONS

Once you open the box your whizzer will look like this.

Everything you need is inside.

Peel the lid off, no screwing needed.



2


Attachments (inc bag, strap) sold separately

A) Remove the white stability ring with the attached funnel

B) then the unisex spout



Ensure the highest part is at the front as a privacy shield.

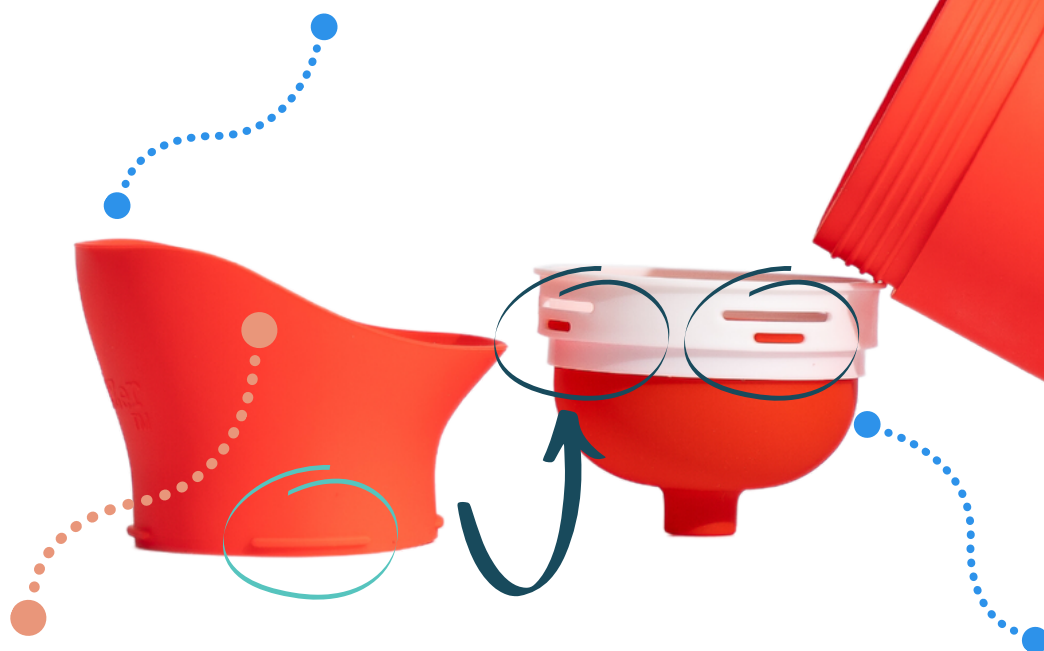

Unisex funnel scoops down to fit beneath the body for females.

The raised bumps on the spout and funnel fit into the holes on the stability ring.

Ensure the parts are secure before use. (it gets quicker after practice)



Leakproof spout can be removed when in use if fast flow fills it too quickly but replace for carrying when full.

4



Either

Once used, remove the spout, stability ring and funnel all in one to empty immediately. Revert steps back to state 1.

Or

Pinch the spout off and replace with the lid and put it all in a bag or use the strap to attach it to something then empty/ clean later.

Attachments (inc bag, strap) sold separately

Tip:
Keep a few
tissues/wipes
with you

We use high-grade food quality silicone so with the right care the Whizzer should last for years of use!


**Clean with hot soapy water after use. *Nothing abrasive. *Dishwasher safe.*

The strap attachment

Attachments (inc bag, strap) sold separately



Fold the strap around the item you're securing it to and slot the ball into the most suitable hole to secure it.

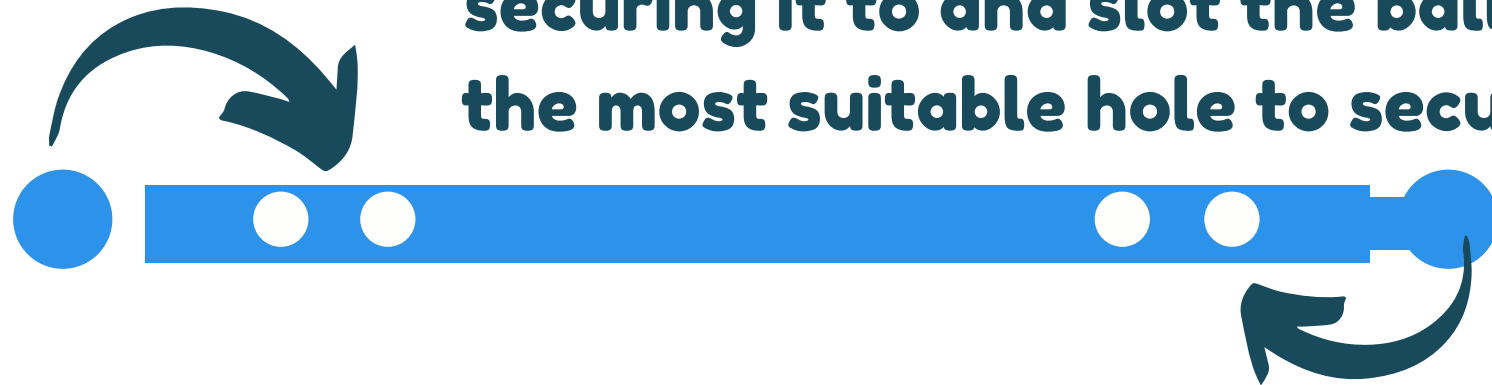

Push the ball through the whizzer side loop and secure into one of the holes

At Salford & Co. we work hard to create a company that works well for everyone involved: our customers, our suppliers, our employees and our planet.

That might sound like a tall order, but we're on a mission and we believe it's possible!



STEP BY STEP INSTRUCTIONS

We wouldn't be anything without our loyal customers. Thank you for being here with us.

1

Once you open the box your whizzer will look like this. Everything you need is inside. Peel the lid off (no screwing needed).

2


Align the spout and bag



The tiny toilet

Packed with innovative features

3

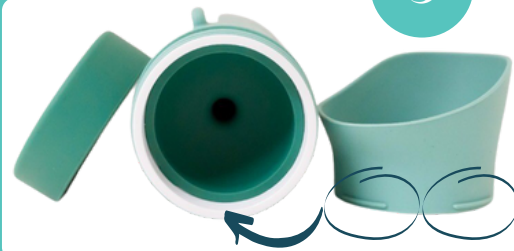

The raised bumps on the spout fit into the holes in the stability ring. Ensure they're lined up/secure before use. (it gets quicker with practice)

4

Ensure the highest part is at the front as a privacy shield.



The spout scoops down to fit underneath the body. Test at home with the child to find a comfortable position.

5



Once used, remove the spout, stability ring and funnel in one to empty immediately. Revert steps back to state 1.

Or pinch the spout off to replace with the lid and put both parts in the bag to empty later.

Tip: Keep a few

