

• ABOUT US •

Driven by the passion for innovation, we at Dr Trust endeavour to provide our customers with the latest medical inventions with an objective to promote good health and wellness all around the world. All the medical devices and health monitors provided by Dr Trust are supported by accurate, latest and ground breaking technologies, innovated at our headquarter in NY, USA. All our products adhere to the most stringent CE and FDA guidelines and are strongly recommended by doctors and health practitioners. Our products are designed in the utmost exemplary ways to ensure that their accuracy and convenience are unrivalled. The ease of their use and operation makes them even more suitable for users of all age groups.

Dr Trust strives to enhance the quality of lifestyle by providing with the most trusted and innovative health care and wellness products. Being a renowned global leader in health care products, Dr Trust ensures that our technically efficient team works dynamically and tirelessly to provide the best of the medical devices to our clients. The products that we have to offer are suitably designed for use at homes, laboratories and hospitals.

Our ground breaking solutions allow you to monitor your health in the easiest ways possible. In today's era when all of our lives are too hassled to handle, it becomes a bit difficult to pay attention to our health. But it has now become easier with the coming of the monitoring devices which can be conveniently used at homes and even on the go.

We bring to you a variety of best self medical devices, trusted and used by Doctors, medical professionals and home users all over the world.

• QUICK STARTUP GUIDE •

Step 1

Loosen all straps from Velcro.

Step 2

Stand straight and wear the brace just like a backpack.

Step 3

Pull your shoulders and adjust the straps to straighten your back.

Step 4

Stick the dual tightening strap to get your comfortable fit.



• INTRODUCTION •

Improving your posture can provide you with the very best in care and support for you and your body. With the Dr Trust Posture Corrector, you can find comfort and relief easily from the all those health problems which are result of a poor posture. The posture corrector helps to support body parts including neck, shoulder, back and make a big difference. We are committed to provide premium quality products so that they can give you the very best in care. We have also included a pair of underarm comfort pads so that you can wear it for a longer period easily.

• WHY DO WE NEED •

Our body does not allow long periods of inactivity and sitting. Many times, we also fail to maintain a proper posture especially while watching TV, standing, walking, and working on a computer. Poor posture problem can be solved easily by using Dr Trust Posture Corrector. The Posture corrector helps in;

- Improving posture
- Reducing back and neck pain
- Reducing BP and increasing blood circulation
- Improving breathing and diaphragm function
- Reducing stress on lower and upper back
- Increasing range of motion
- Improving nerve signals from the spine
- Reducing fatigue
- Promoting proper growth development in children
- Improves herniated discs
- Reducing pressure to internal organs and many others

• HOW TO USE •

Putting on & adjusting Dr Trust Posture Corrector

1. Adjust back strap, if needed



2. Loosen straps by lifting bottom of each buckle and pulling down on the nylon strap that connects to back of corrector.



Put on Posture Corrector Using One of Two Methods

3. Put on like a backpack-over one shoulder, behind the back, then over the second shoulder.



Or

Grasp straps with both hands and lift overhead, allow corrector to slide down arms, fall behind head & settle over shoulders.



4. Pull down on nylon straps to tighten. If strap bunches, pull down to straighten.




5. Tuck the loose straps beneath bands to keep from dangling.



• ADDING COMFORT PADS •

1. Before tightening straps, slide each pad under the shoulder straps, and position where comfortable.

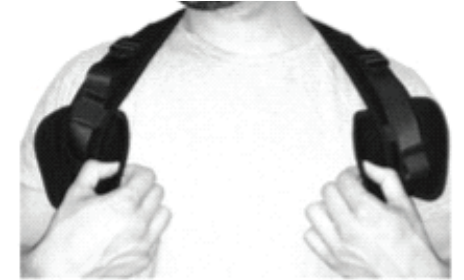

2. Once pads are in place, tighten straps for snug fit.



3. Wrap each pad around strap and press it close to keep in place.



CARE AND MAINTENANCE

- Gentle or delicate machine wash
- Hand wash
- Warm water

CAUTIONS

- Do not tumble dry
- Do not iron

SPECIFICATIONS

Posture Corrector Material	75% neoprene, 25% nylon
Size range	27"-47"
Product dimension	28*32 cm
Weight	200 g
Net weight	0.165 kg

CUSTOMER SUPPORT

CONTACT ADDRESS

USA

Nureca INC.USA
276 5th Avenue, Suite 704-397,
New York (NY) - 10001, USA

INDIA

Corporate Office (Mumbai)

Nureca Limited
Office No. 101, 1st Floor, Udyog Bhavan, Sonawala Lane,
Goregaon East, Mumbai, Maharashtra - 400063.

Contact us

India: +91-7527013265 / +91-9356658436

Website: www.drtrust.in

Corp Website: www.nureca.com

Email: customercare@nureca.com

Connect with us on social networks

Facebook: @drtrust

Instagram: @drtrustisin

Youtube: NurecaUsa

COPYRIGHT © 2022 NURECA LTD ALL RIGHTS RESERVED



Scan to View
Product Demo Video
www.drtrustusa.com/313